R4750, R4751
R4752, R4780
R4781, R4782

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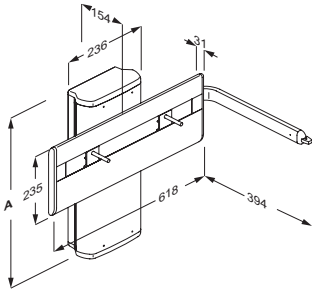
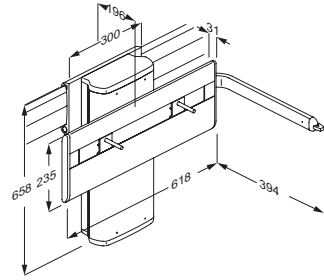
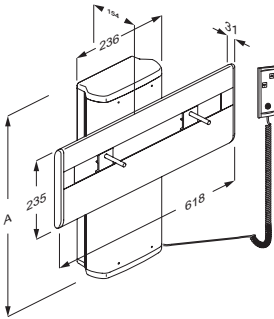
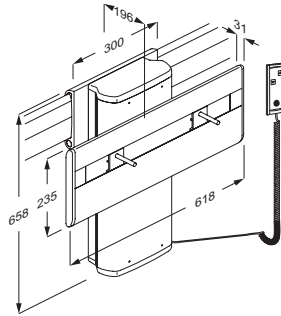
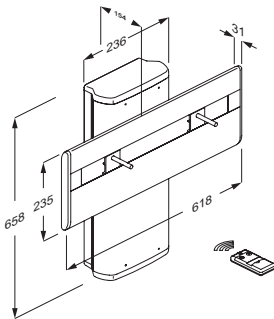
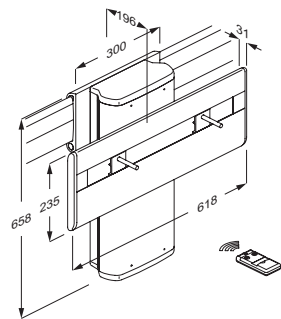
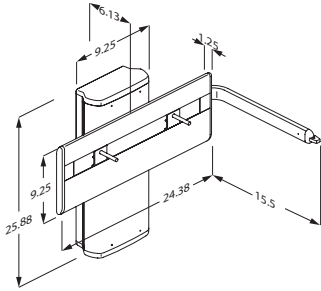
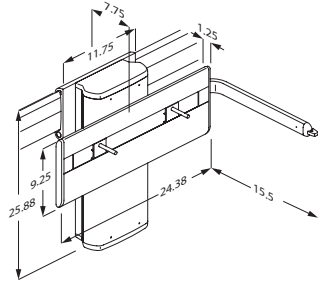
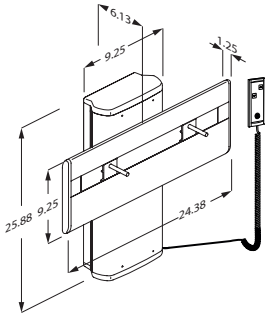
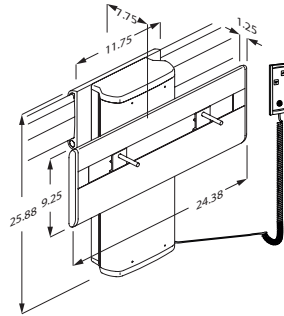
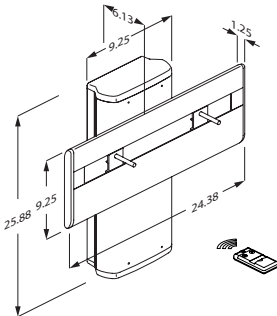
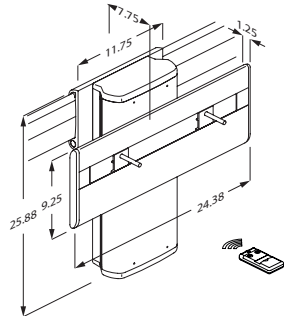
R4782

Content

Mounting instruction	6
Operational signage	6
Symbols used in this manual	7
General safety	8
Product label	9
Intended use	11
Intended operator profile	11
Operating environment	11
Operating instructions	12
Height adjustment	12
Sideways adjustment - R4780, R4781 and R4782	13
Cleaning	14
To remove limescale	14
Maintenance	15
Service inspection	15
Trouble-shooting	17
Technical data	17
Disposal and recycling	18

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressaliticare.com.

Operational signage

An operational signage can be downloaded from www.pressaliticare.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Electricity or potentially dangerous voltage.

Head injury warning.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

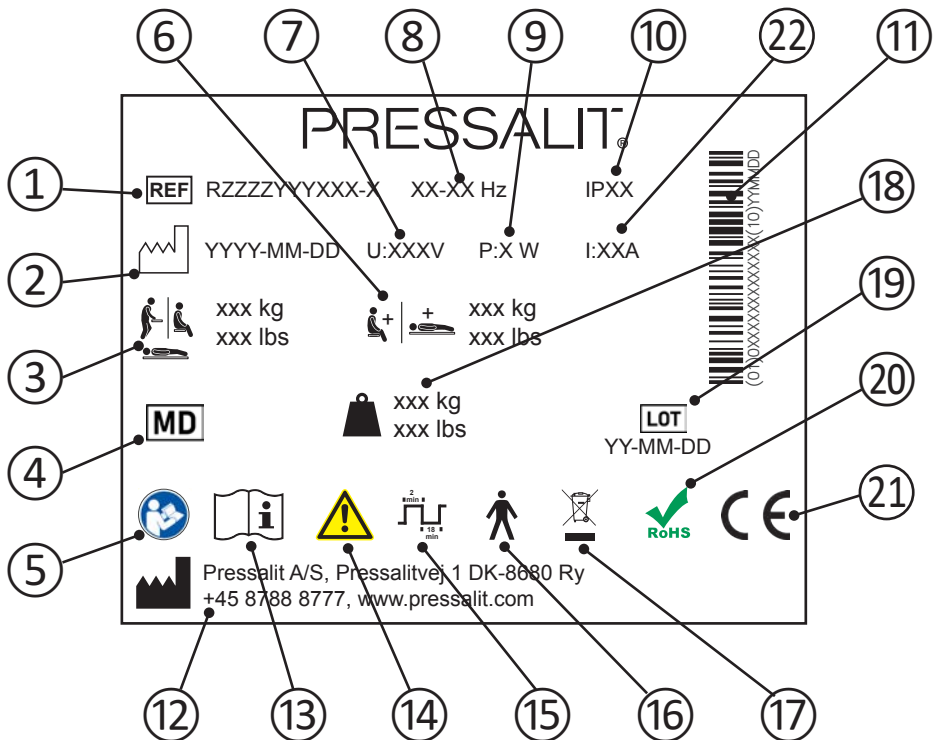
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

- ① Article number

- ② Production date

- ③ Maximum user weight

- ④ Medical device

- ⑤ Consult manual for use

- ⑥ Max load

- ⑦ Mains voltage

- ⑧ Frequency

- ⑨ Maximum power consumption

⑩ IP class

⑪ GS1-128 barcode

⑫ Name and address of manufacturer

⑬ Consult manual for use

⑭ Consult the manual for important cautionary information such as warnings and precautions

⑮ Duty cycle

⑯ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

⑰ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑱ Product weight

⑲ Batch code

⑳ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

These products are CE-marked in accordance with relevant requirements in:

- ㉑
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at www.pressalit.com.

㉒ Electric Current

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

Operating instructions

The wash basin bracket can be adjusted 300 mm / 11 6/8 inches up and down.
Max load is 100 kg / 220 lbs.

Take care to avoid injury to hands when in the vicinity of closing moveable parts.

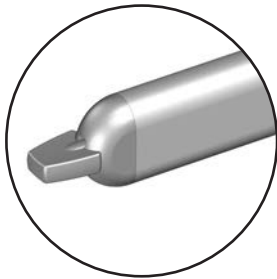
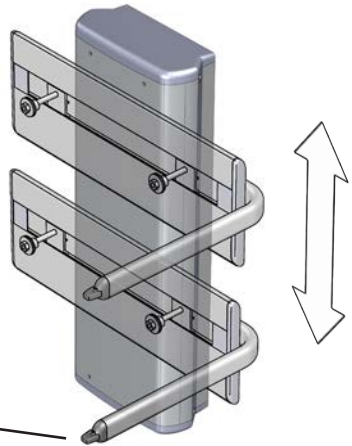
Take care to avoid injury to your head when in the vicinity of moving mechanical parts.

Height adjustment

R4750 and R4780

The height adjustment is controlled by the lever.


Press and hold the lever up to raise the wash basin, and press and hold the lever down to lower the basin. When the lever is released, the motion stops.

R4751 and R4781

The height adjustment is controlled by the wired hand control unit.

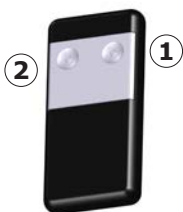Press and hold the Up button ① to raise the wash basin bracket, and press and hold the Down button ② to lower the bracket. When the button is released, the motion stops.

R4752 and R4782

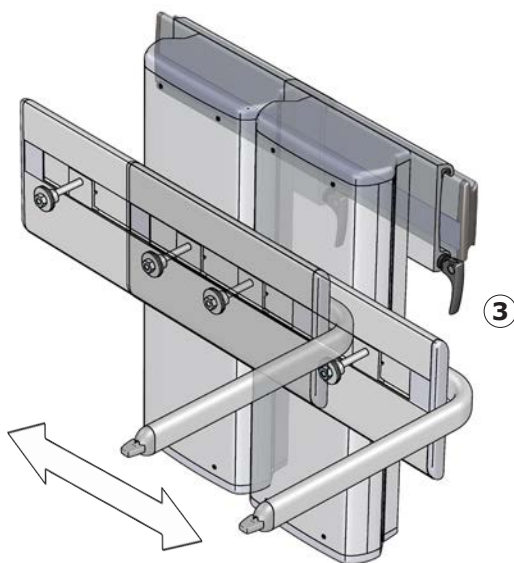
The height adjustment is controlled by the cordless remote control.

Press and hold the Up button ① to raise the wash basin bracket, and press and hold the Down button ② to lower the bracket. When the button is released, the motion stops.

Never place objects underneath the wash basin.

Sideways adjustment - R4780, R4781 and R4782

Release the locking handle ③ and move the wash basin bracket sideways. Tighten the locking handle again to secure the bracket in place.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on www.pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that service inspection is done once a year.

Service inspection

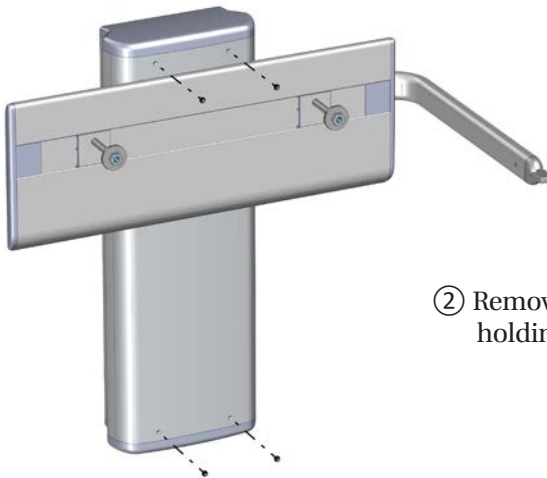
Take the wash basin to its highest position.

Disconnect mains before removing the covers.


① Loosen the two top screws ①.


② Remove the four screws holding the front cover.

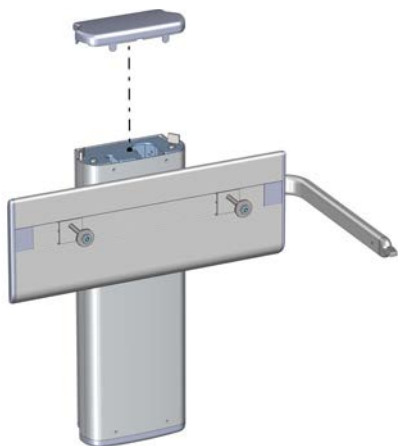en


③ Lift off the top cover.

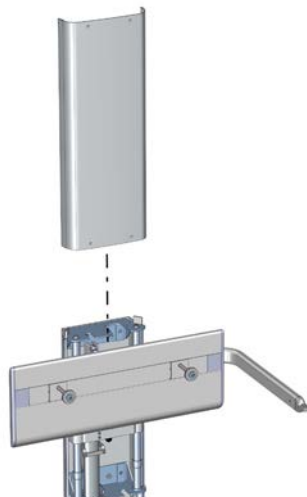④ Remove the front cover by sliding it upwards.

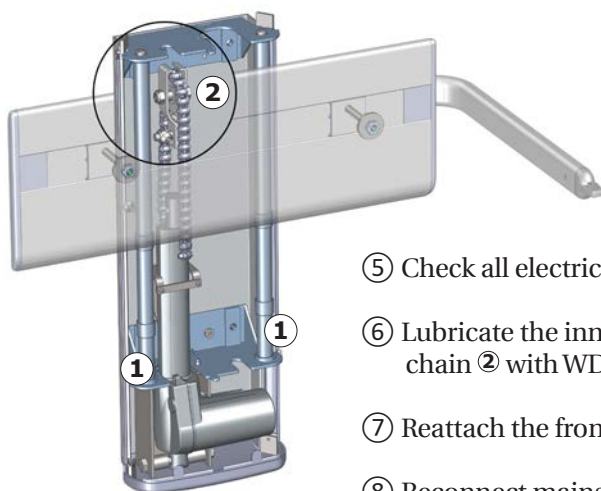en

⑤ Check all electrical connections.

⑥ Lubricate the inner tubes ① and the chain ② with WD-40 or equivalent.

⑦ Reattach the front and the top cover.

⑧ Reconnect mains.

⑨ Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Trouble-shooting

Symptom	Cause	Action
The product will not work, even though the controls is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data

Max load	100 kg / 220 lbs
Materials	Aluminium Plastics
Weight	R4750: 16,36 kg / 36,06 lbs
	R4751: 15,94 kg / 35,14 lbs
	R4752: 16,08 kg / 35,45 lbs
	R4780: 18,12 kg / 39,94 lbs
	R4781: 17,70 kg / 39,02 lbs
	R4782: 17,84 kg / 39,33 lbs
IP class	IPX6
Power specifications	
Max consumption	75 W
Voltage	Mains connection: 230V/120V/100V
Frequency	50/60 Hz
Current	1.6 A

en

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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