R9502717
R9502719
R9502721

MCT 5

Mobile change trolley – Operation and maintenance manual	en
Mobilt skifteleje – Brugs- og vedligeholdelsesvejledning	dk
Mobile Ankleideliege – Gebrauchs- und Pflegeanleitung	de
Chariot de change mobile – Manuel d'utilisation et d'entretien	fr
Mobiele aankleedtafel – Gebruiks- en onderhoudshandleiding	nl
Mobil vårdbräns – Bruks- och underhållsanvisning	sv
Mobilt stellebord – Bruks- og vedlikeholdsanvisning	no
Camilla móvil para cambio – Instrucciones de uso y mantenimiento	es
Barella mobile cambio indumenti – Manuale d'uso e manutenzione	it
Vaatteidenvaihtovaunu – käyttö- ja huolto-opas	fi
Wózek toaletowy – Instrukcja obsługi i konserwacji	pl

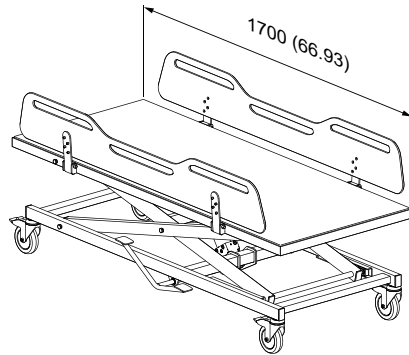
English	8
Dansk	24
Deutsch	40
Français	56
Nederlands	72
Svenska	88
Norsk	104
Español	120
Italiano	136
Suomi	152
Jzyk polski	168


Pressalit's Environmental and Quality Management Systems are certified by LRQA

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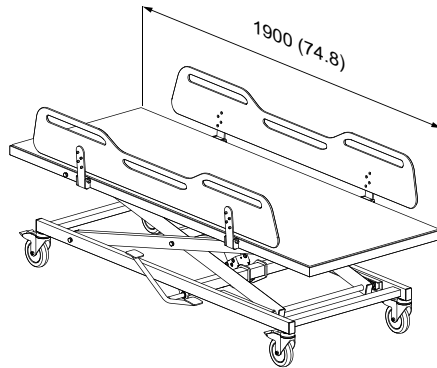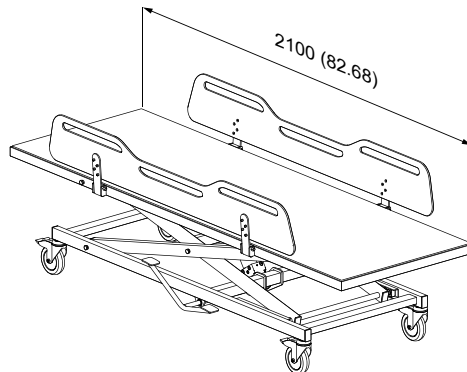
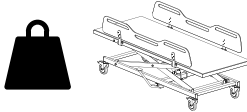
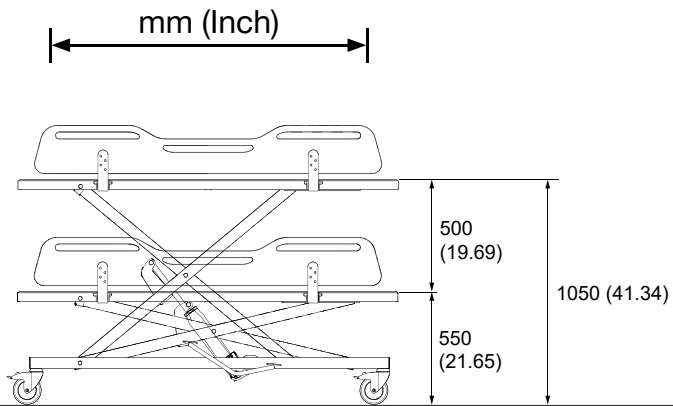
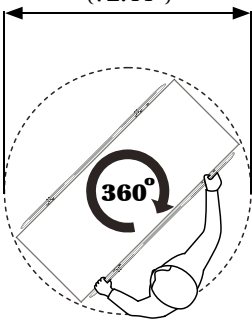
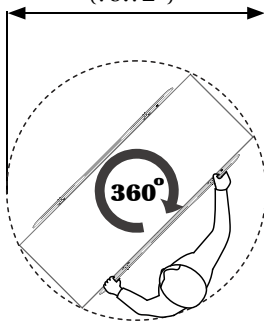
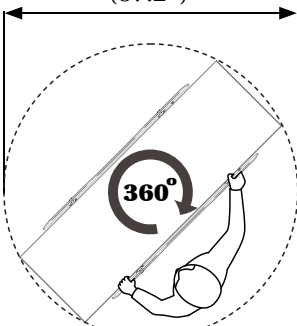
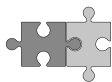
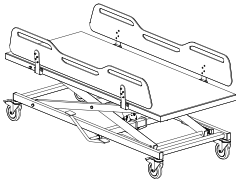
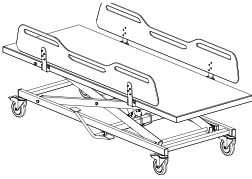
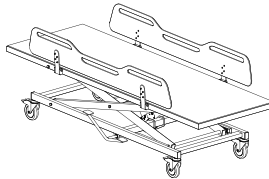
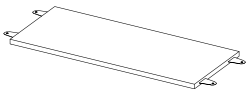
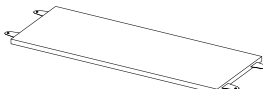
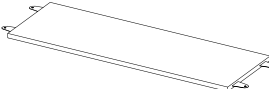
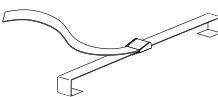
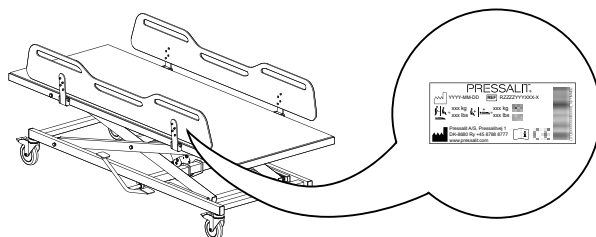
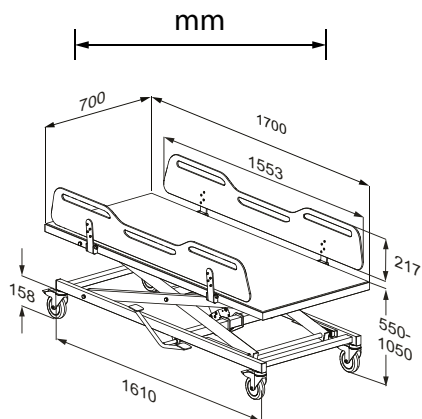
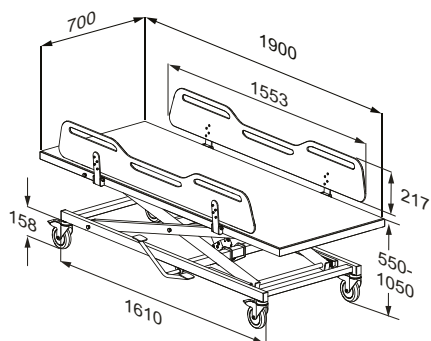
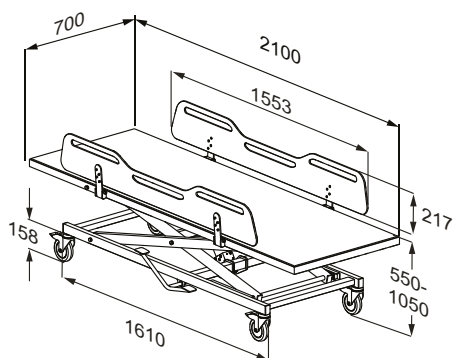
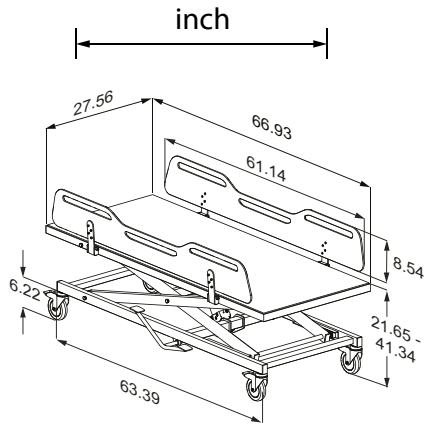
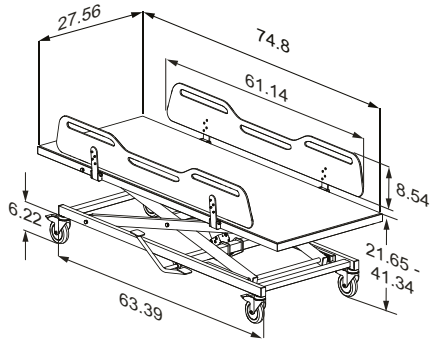
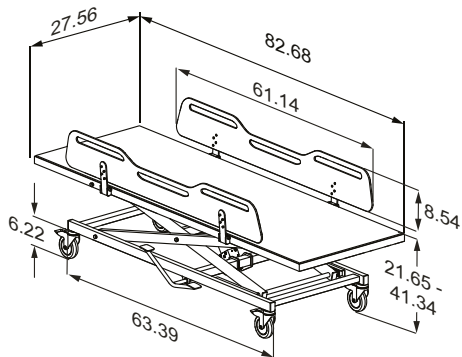
R9502721

Content

Symbols used in this manual	9
General safety	10
Safety distances	11
Intended use	12
Intended operator profile	12
Intended user profile	12
Operating environment	12
Contraindications for use	13
Biocompatibility	13
Flammability	13
Operating instructions	14
Safety brakes	14
Safety rails	15
Height adjustment	16
Moving	17
Accessories	18
Mattress	18
Body belt	18
Cleaning	19
Disinfection	19
Maintenance	20
Maintenance inspection	20
Product label	21
Storage and transportation	22
Troubleshooting	22
Technical data	23
Disposal and recycling	23

en

Operational signage

A Quick-guide is included with the product. It can also be downloaded from pressalit.com.


Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

	<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>
	<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>
	<p>IMPORTANT indicates important information about handling and use of the product.</p>
	<p>NOTICE is used to address practices not related to personal safety.</p>

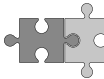en

 <p>Head injury warning</p>	 <p>Hand injury warning</p>	 <p>Genitalia injury warning</p>	 <p>Feet injury warning</p>
 <p>Spare parts</p>	 <p>Do not leave patient unattended</p>	 <p>Compatible products</p>	 <p>Product weight</p>
 <p>Max. user weight</p>	 <p>Article number / versions</p>		

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

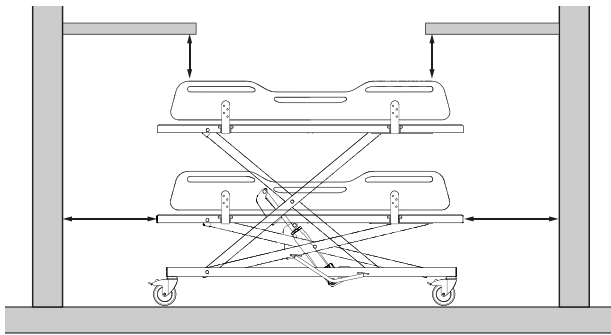
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

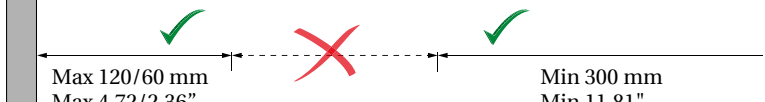
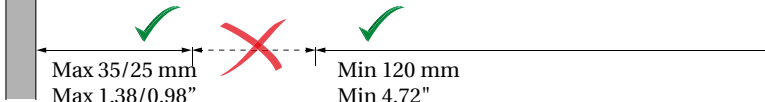
NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Safety distances

Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

ISO 17966:2016			
		Max 120/60 mm Max 4.72/2.36"	Min 300 mm Min 11.81"
		Max 8/4 mm Max 0.31/0.16"	Min 25 mm Min 0.98"
		Max 8 mm Max 0.31"	Min 75 mm Min 2.95"
		Max 35/25 mm Max 1.38/0.98"	Min 120 mm Min 4.72"

WARNING

If the unit moves within the safety distances, you must keep a close eye on especially surrounding people and objects to avoid crushing hazards.

en

Intended use

This product is intended for changing a person who is unable to stand or sit. The person is lying horizontally on the change trolley.

Intended operator profile

Only personnel who are trained in healthcare and/or in changing someone under their care should use this product.

Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

The change trolley is intended to be used at room temperature in public changing places, schools, private homes, day/residential/nursing facilities and hospitals under normal working conditions.

The change trolley may be used in dry rooms and not as a shower table in showers. If the relative humidity is above 70%, the expected life of the unit will be significantly reduced. The change trolley must not be exposed to chlorine-containing fumes.

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Contraindications for use

If the product is being used for users with damaged skin, wounds, or other similar conditions, it is recommended to use a towel as a barrier to prevent open wounds from coming into direct contact with the substrate.

Biocompatibility

The biocompatibility of the materials that come in contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involves assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use.

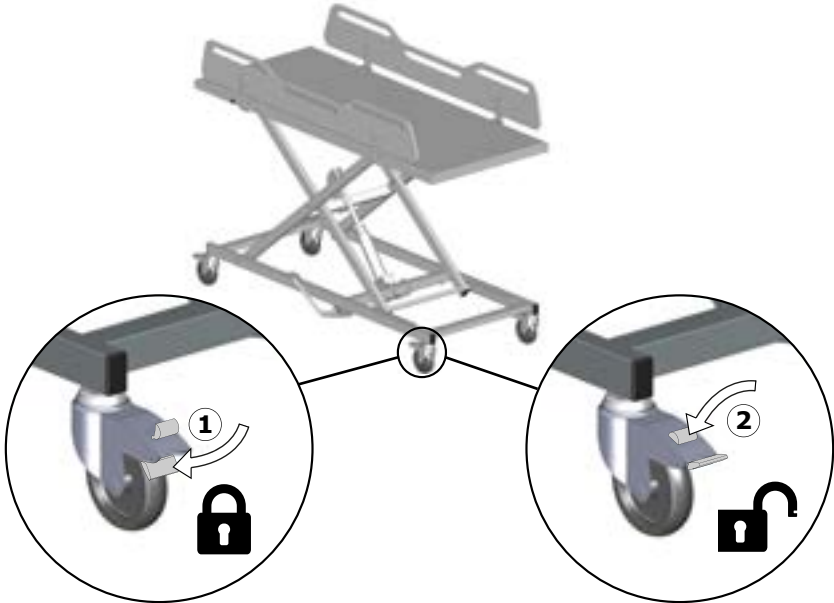
Flammability

The product materials have undergone flammability testing in accordance with ISO 8191-1:1987 and ISO 8191-2:1988.

Operating instructions

Safety brakes

The change trolley is equipped with brakes on all four wheels to prevent accidental movement.

en

Lock the brakes①: Step on the lock pedal on all four wheels.

Unlock the brakes②: Step on the release button on all four wheels.

Always use your foot to lock/unlock the wheels. Wear closed-toe shoes and avoid using your hands. Be mindful of the brakes to avoid tripping or bumping into them.

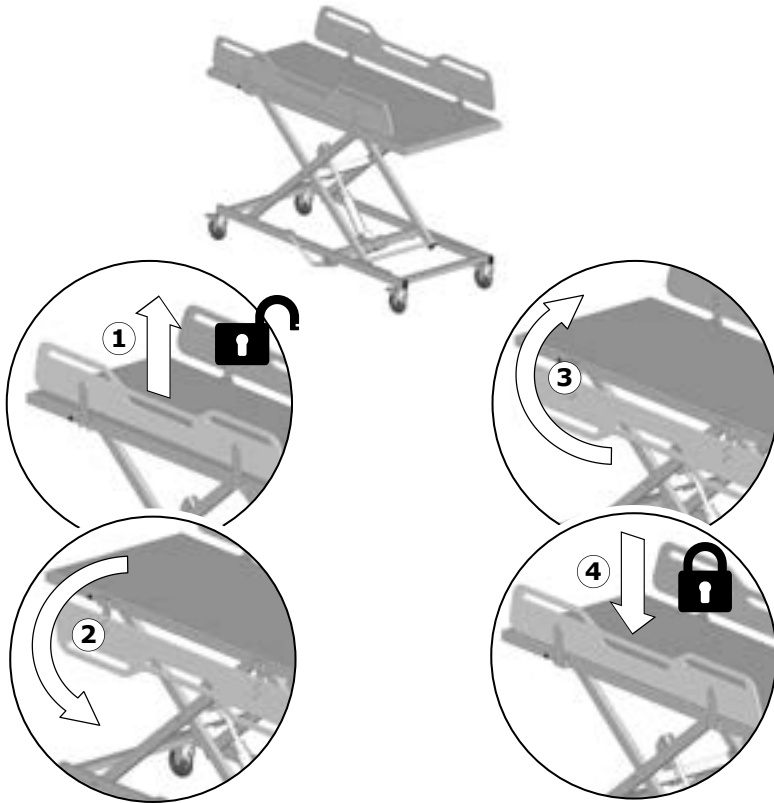
Ensure all four wheels are locked before transferring a patient to/from the mobile change trolley.

Never leave a patient unattended on the trolley.

Safety rails

The safety rails can be folded up and down individually.

Fold down: Lift the safety rail upwards to unlock it①, and swing it down②.

Fold up: Swing the safety rail up③ and push it down to lock it in place④.

CAUTION

To protect the patient, it is recommended that the side rails are upright and locked when using the mobile change trolley.

Be careful when adjusting the side rails to avoid pinching hands, feet, or other body parts.

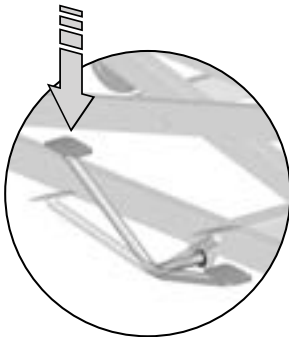
Height adjustment


200 kg / 440 lbs

The mobile change trolley can be adjusted 500 mm / 19.69" up and down. The maximum height is 1050 mm / 41.34".

The change trolley features a double foot pedal for adjusting its height.

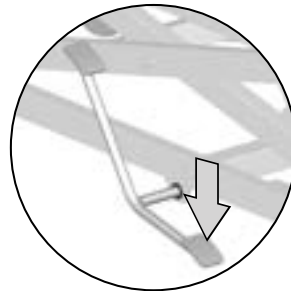Step repeatedly on the long pedal to raise the unit.

Depress the short pedal to lower the unit.

Do not place your foot on the bearing frame and be especially cautious of the risk of pinching hands, feet, or objects when adjusting the height of the unit.

! WARNING

Never let children or pets be near the unit during height adjustment.

Moving

To move the trolley, unlock all four wheels. Grab the lying surface itself, not the safety rails, and proceed with caution while moving the unit.

en

To prevent the trolley from tipping over, avoid using it on floors with lowered drains, holes, or significant slopes.

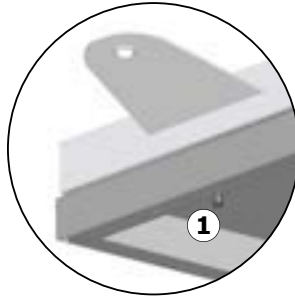
Accessories

A mounting instruction is included with the accessories and can also be downloaded from pressalit.com.


Mattress

A mattress can be added to the change trolley for increased comfort. The mattress is reversible and can be used on both sides.

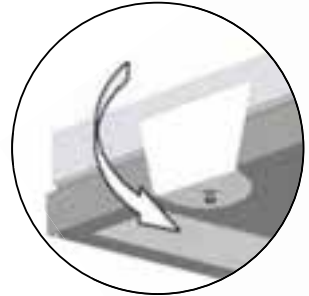To mount the mattress, loosen the locking screw^① by about one centimeter.



Pull the mattress flap down over the screw. Repeat this with all four flaps.

Body belt

en

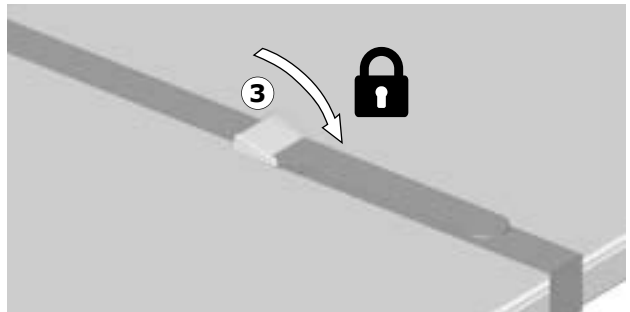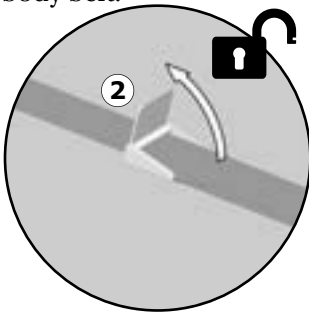To provide the user with additional support and comfort, you can install a body belt.

Pull the handle on the buckle up^② to release the lock. Then press the handle down again to fasten the body belt^③.

The body belt does not provide safety and should never replace supervision.

Ensure the belt is securely fastened and does not come near the patient's neck, as it poses a strangulation risk.

IMPORTANT

To maintain hygiene, it is recommended to check for any visible damages or signs of wear and tear on the mattress and the body belt regularly. If any damages are found, the mattress and the body belt should be replaced immediately to prevent any risk of contamination or infection.

Cleaning

To clean the product, use a damp cloth that has been wrung out in a mild soap mixture, such as universal cleaning solution.

After cleaning, it is essential to wipe the product with a dry, soft cloth. This step is crucial to prevent the materials from discoloration. Additionally, it helps prevent any potential degradation of the surface paint.

A list of suitable cleaning agents/disinfectants tested by Pressalit can be found in the technical datasheet or on pressalit.com under the specific product.

CAUTION

- Always read the supplied instructions of the cleaning agent and comply with these instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.
- Do not perform any type of cleaning or disinfection of the change trolley while the user is lying on it.

IMPORTANT

- Do not rinse or spray water directly on the product.
- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acid or base.
- Avoid using abrasive, caustic or chlorine-based cleaning agents

Disinfection

When using the mobile change trolley for multiple patients, it is necessary to both clean and disinfect it between uses on different patients to prevent the risk of cross infection.

Maintenance

Maintenance inspection should be carried out once a year. The inspection should be recorded in a logbook describing what has been checked and replaced. Spare parts should always be obtained from your supplier or Pressalit in order to ensure a continuously safe product and the guarantee. Contact your supplier or Pressalit for purchase of spare parts and drawings. Perform a visual inspection of steel parts for penetrating rust.

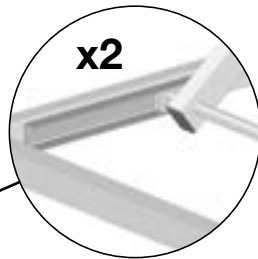
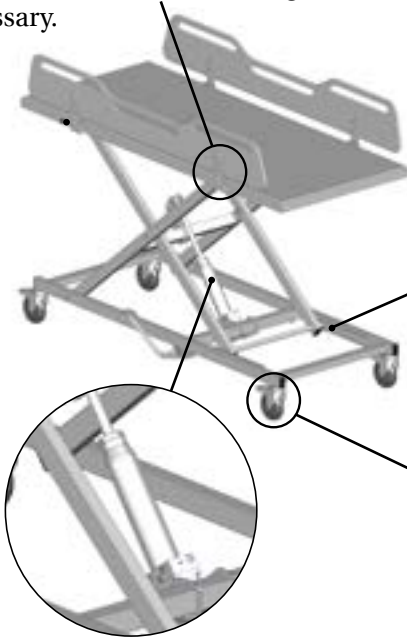
Maintenance inspection


Check whether all nuts, screws and hinges have become loose. Tighten if necessary.

NOTICE


If the product is used in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.

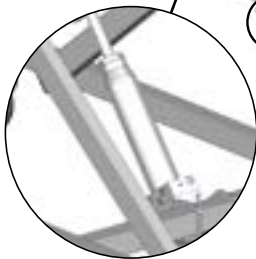Lubricate the sliding rails at least every six months. Use a food grade grease.

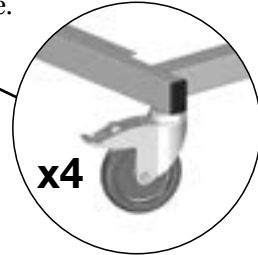
Inspect the hydraulic pump for any signs of oil leaks. If minor leaks are detected, they can be cleaned with a dry cloth.

Check the trolley and wheels for corrosion. Replace the wheels if necessary.

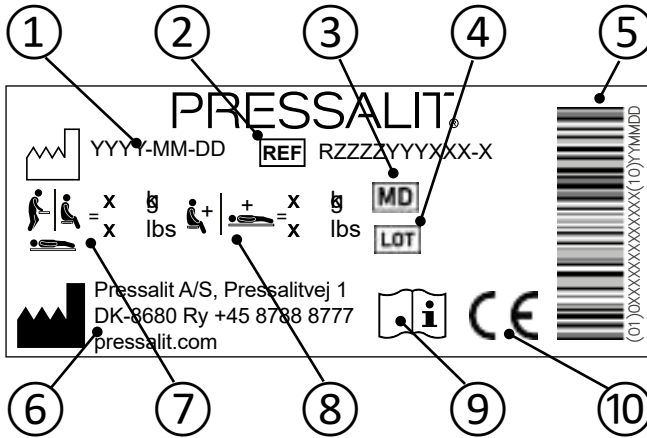Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

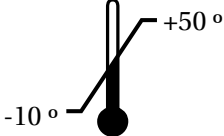
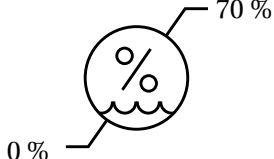
⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at pressalit.com.

en

Storage and transportation

	<p>The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius (14 and 122 degrees Fahrenheit).</p>
	<p>Keep humidity below 70%.</p>

IMPORTANT

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

en

Troubleshooting

Symptom	Cause	Action
The trolley has difficulty moving up and/or down.	The lifting unit lacks lubrication.	Lubricate the sliding rails. See Maintenance .

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the warranty. Expected service life of the shower change trolley is 10 years.

Technical data

Max. user weight	200 kg / 440 lbs	
Materials	Powder-lacquered steel Plastics CDF	
Weight	R9502717	82 kg / 180 lbs
	R9502719	84 kg / 185 lbs
	R9502721	86 kg / 190 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected. When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion. The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available. If such systems are not available, incineration of plastic waste can be an alternative, as it can generate energy that can be used for power production.

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