




**Maximum load**

= **200 kg**  
 = **440 lbs**




Never leave a person on the table unattended.

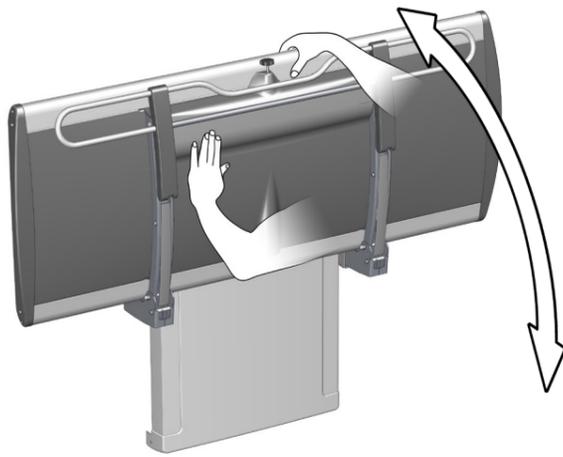

**Folding the table up and down**


Support the table using your hand or body when folding it up and down, to avoid the risk of damage. Fold the table up against the wall after use.

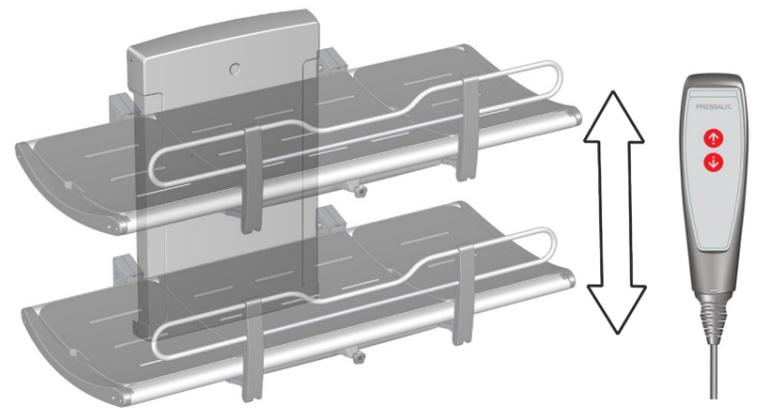


**Height adjustment**

Press and hold the Up button to raise the table. Press and hold the Down button to lower the table.


**① To pull the safety rail up**

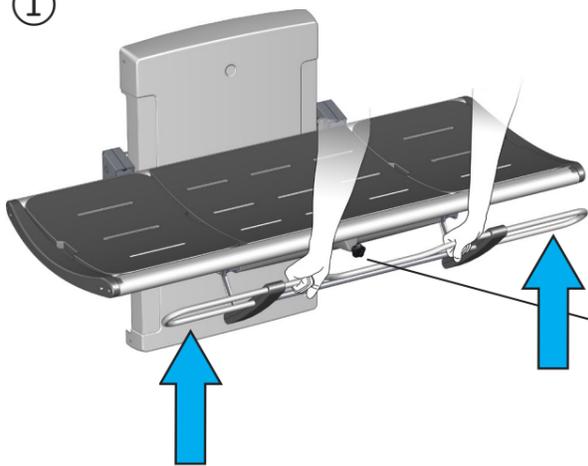


Turn the locking knob one way or another to release the safety rail. Lift the safety rail upwards until you hear a click indicating that it is locked.

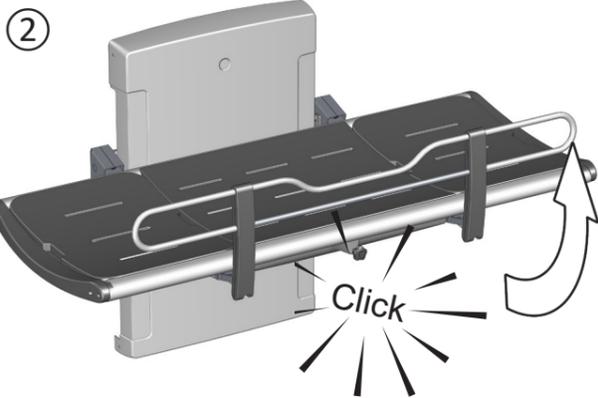

**To fold the safety rail away**

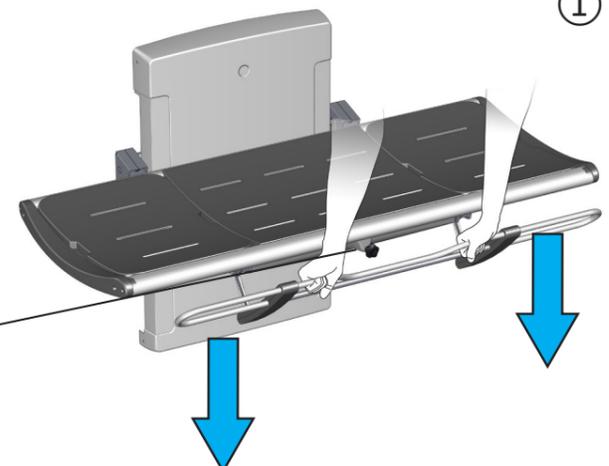


Turn the locking knob one way or another to release the safety rail. Push the safety rail back to the underside of the table until you hear a click indicating that it is locked.

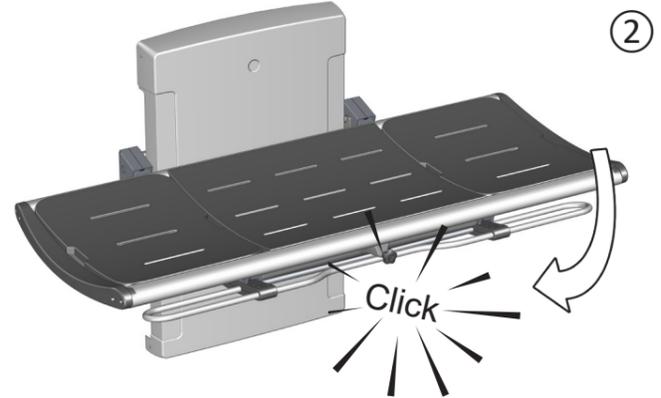


**Head support**

Use the cut-out grip to lift the pad for head support. Fold the metal bracket down to hold the pad in the upright position.

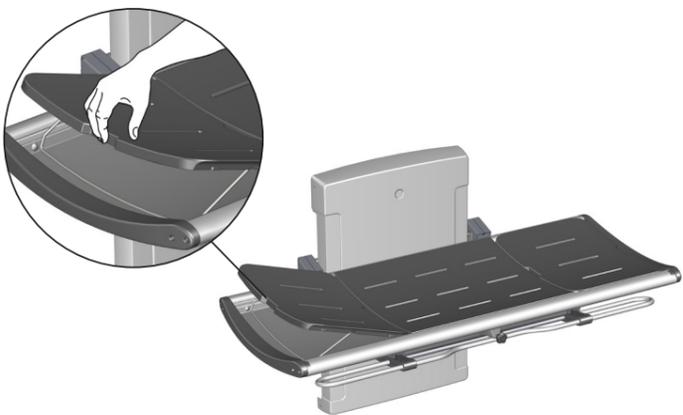


**Cleaning**

Clean the table with sanitizing wipes provided after use and dispose of wipe in the bin. Use the cut-out grips when lifting the pads.

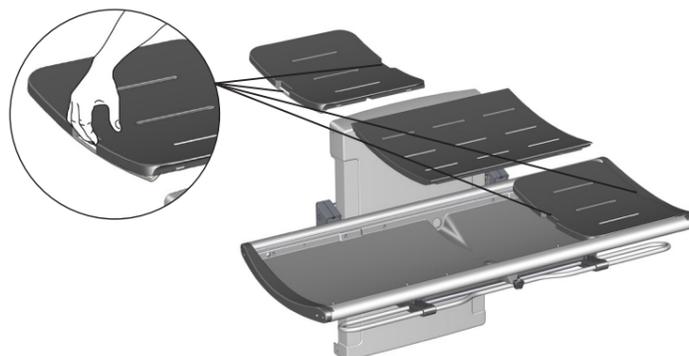

Never let a person lie on the shower table without the pads in place. The tray under the pads is for water collection only. It is not to be used as a bathtub. Risk of user injury and damage to product.

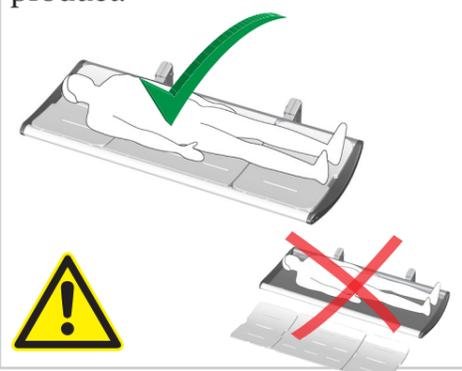

This space is intentionally left blank to make room for Changing Place operator contact details.