R8711 - R8712 - R8713
R8911 - R8912 - R8913
R8721 - R8722 - R8723
R8921 - R8922
R8732 - R8733
R8734 - R8735

Day care changing table – Operation and maintenance manual	en
Puslebord – Brugs- og vedligeholdelsesvejledning	dk
Wickeltische – Gebrauchs- und Pflegeanleitung	de
Tables à langer – Manuel d'utilisation et d'entretien	fr
Aankleedtafels – Gebruiks- en onderhoudshandleiding	nl
Skötbord – Bruks- och underhållsanvisning	sv
Stellebord – Bruks- og vedlikeholdsveiledning	no
Hoitopöytä päivähöitoon – käyttö- ja huolto-opas	fi
Cambiador - Instrucciones de uso y mantenimiento	es
Przewijak – Instrukcja obsługi i konserwacji	pl
换洗床 - 操作和维护手册	中文

English	14
Dansk	27
Deutsch	40
Français	53
Nederlands	66
Svenska	79
Norsk	92
Suomi	105
Español	118
Polski	131
中文	144

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


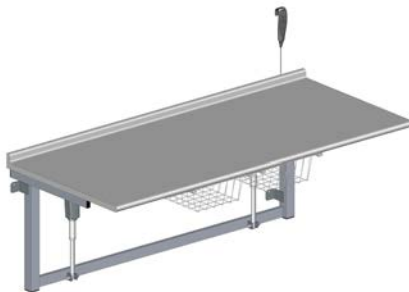
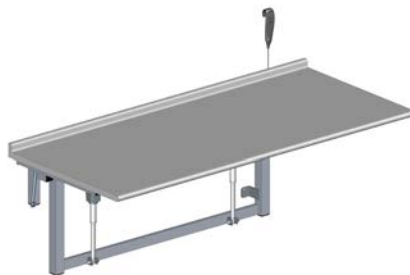
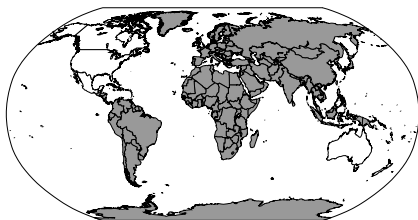
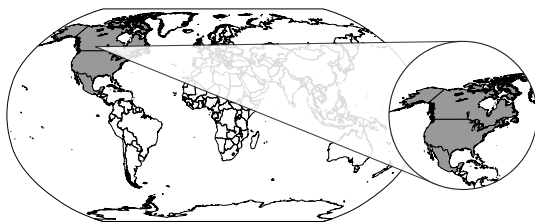
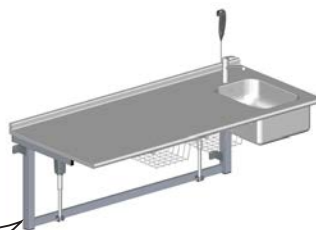
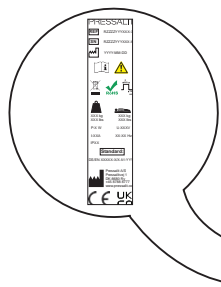
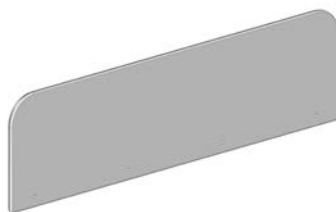
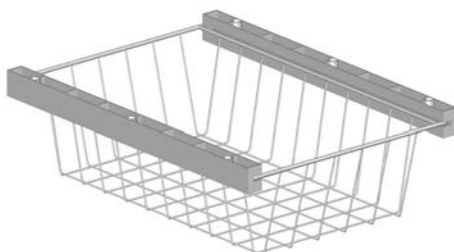
	Max P	U	Max I
R8711, R8712, R8721 R8722, R8732	75W	230V	0.3
R8911, R8912 R8921, R8922	75W	120V	0.7
R8713, R8723 R8733, R8735	150W	230V	0.7
R8913	150W	120V	1.4

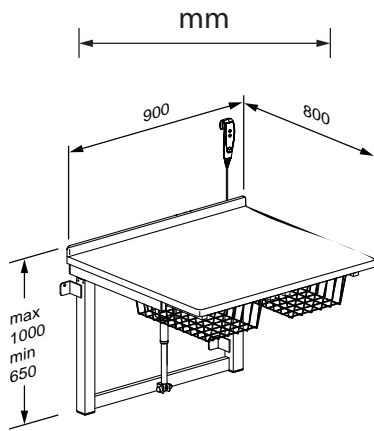
R8711, R8911

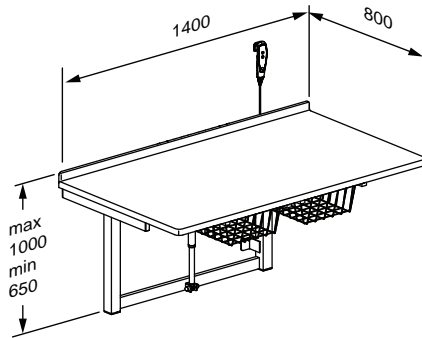
R8712, R8912

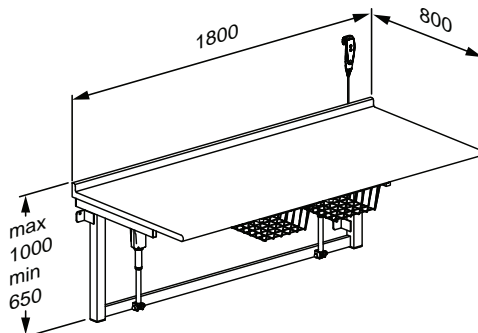
R8713, R8913

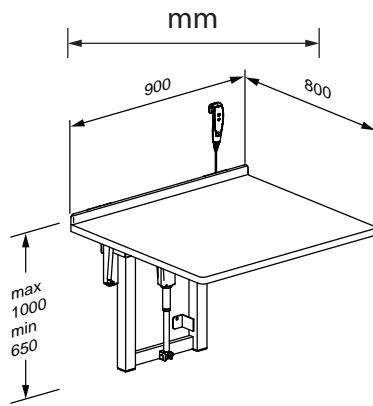
R8721, R8921

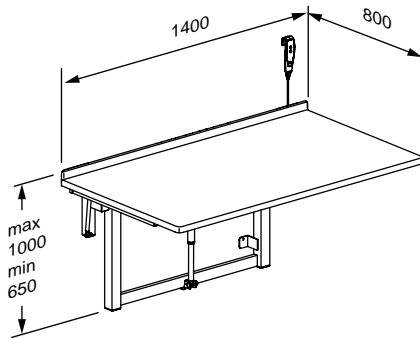
R8722, R8922

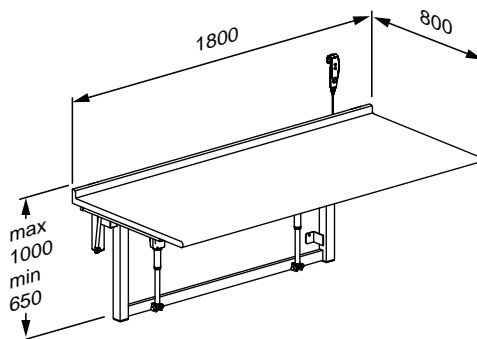
R8723

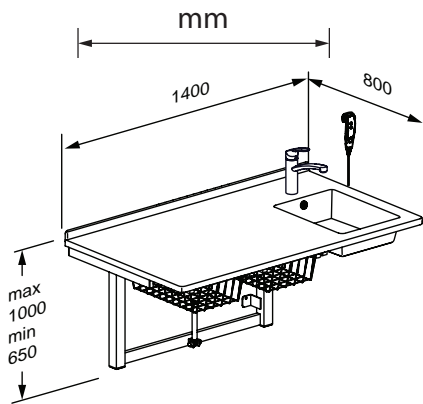
R8732, R8734

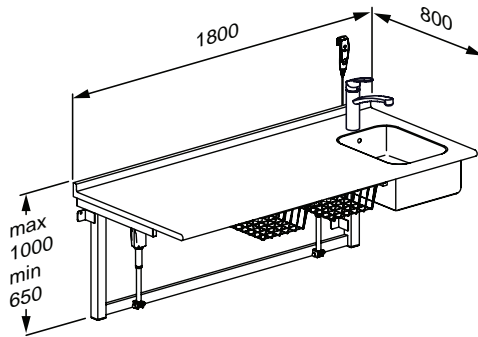
R8733, R8735

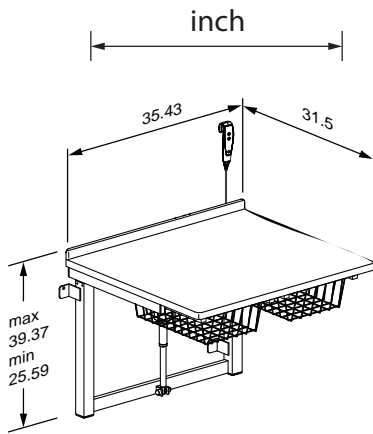
R8711, R8911

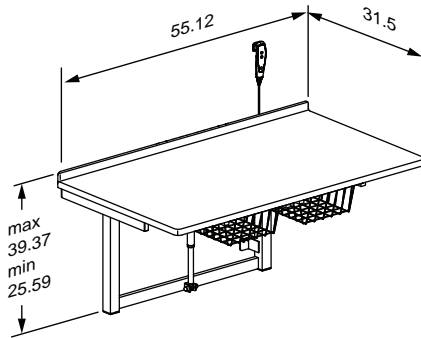
R8712, R8912

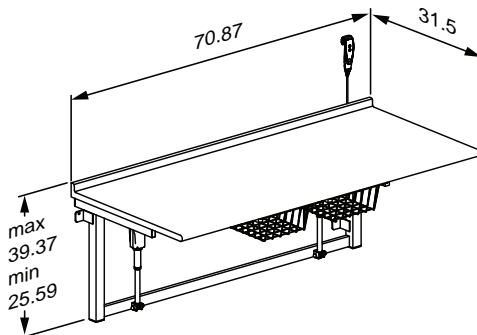
R8713, R8913

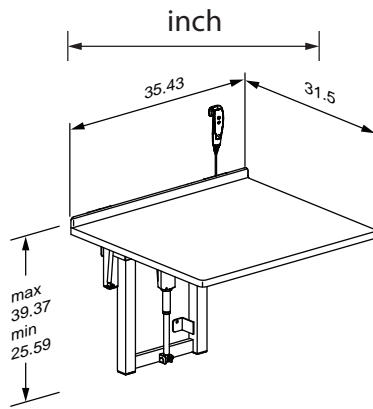R8721, R8921

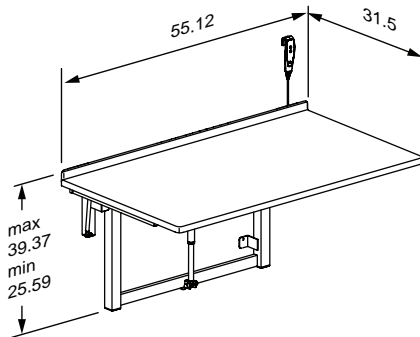R8722, R8922

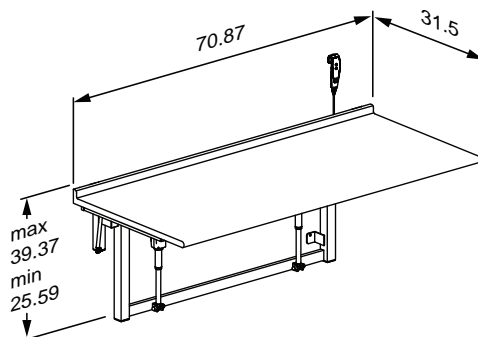R8723

Content

Mounting instruction	14
Operational signage	14
Symbols used in this manual	15
General safety	16
Safety distances	17
Intended use	18
Intended operator profile	18
Operating environment	18
Operating instructions	19
Folding up and down	20
Cleaning	21
Maintenance	22
Maintenance inspection	22
Product label	23
Trouble-shooting	25
Storage and transportation	26
Disposal and recycling	26

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalit.com.

IMPORTANT

RETAIN THIS MANUAL FOR FUTURE REFERENCE. READ CAREFULLY.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING		<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>	
 CAUTION		<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>	
IMPORTANT		<p>IMPORTANT indicates important information about handling and use of the product.</p>	
NOTICE		<p>NOTICE is used to address practices not related to personal safety.</p>	
 <p>Head injury warning</p>	 <p>Hand injury warning</p>	 <p>Feet injury warning</p>	 <p>Electricity / voltage warning</p>
 <p>Strong heat warning</p>	 <p>Do not leave child unattended</p>	 <p>Compatible products</p>	 <p>Spare parts</p>
 <p>Power specifications</p>	 <p>Product weight</p>	 <p>Max. user weight</p>	 <p>Article number / versions</p>

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

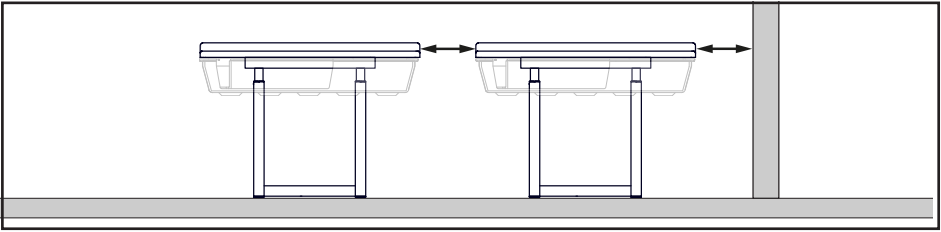
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.




Safety distances

Make sure that the changing table is at safe distance from surrounding objects. Check the distances before any use.

	EN 12221-1:2008	Pressalit recommends
	Max 65 mm (2.56") Min 223 mm (8.78")	Max 60 mm (2.36") Min 300 mm (11.81")
	Max 7 mm (0.28") Min 12 mm (0.47")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 25 mm (0.98") Min 45 mm (1.77")	Max 8 mm (0.31") Min 120 mm (4.72")

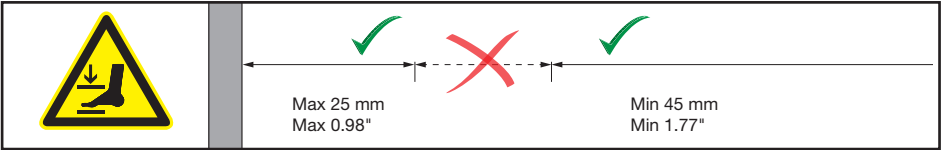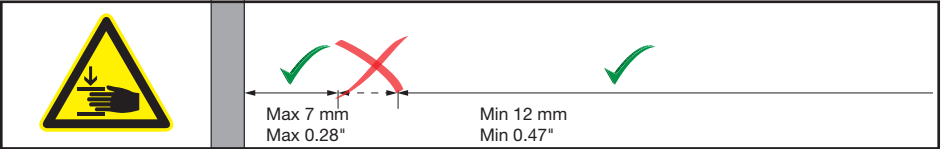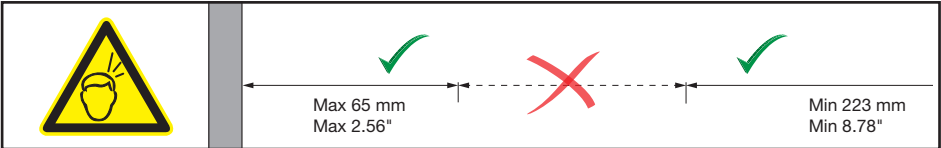en

Intended use

This product is intended for optimum ergonomics when caring for children.

When using a changing table with one actuator, the weight of the child must not exceed 75 kg / 165 lbs.

When using a changing table with two actuators, the weight of the child must not exceed 150 kg / 330 lbs.

Intended operator profile

The product shall only be operated by adults.

Operating environment

The product is intended to be used indoor at room temperature under normal working conditions.

Life cycle test performed according to ISO 17966:2016.

This product is tested according to DS/EN 12221-1 and 12221-2.

en

Do not leave the child unattended.

Be aware of the risk of open fires and other sources of strong heat, such as electric bar fires, gas fires, etc. in the near vicinity of the changing table.

CAUTION

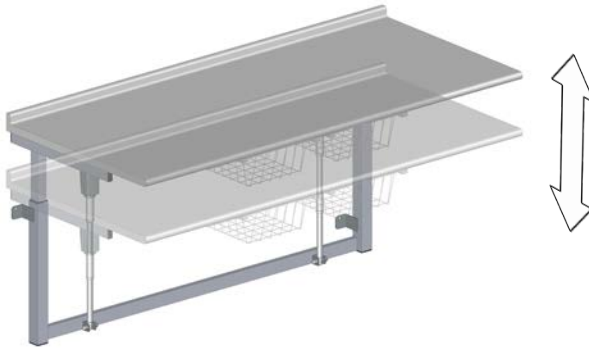
Never allow children to operate this product without supervision.

NOTICE

Always place the child on a changing table mattress and not directly on the table.

Operating instructions

The changing table can be adjusted from 650 to 1000 mm / 25 to 39 inches up and down.

The height adjustment is controlled by the wired remote control unit.

Press and hold the up button to raise the unit. When the button is released, the motion stops.

Press and hold the down button to lower the unit. When the button is released, the motion stops.

en

To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

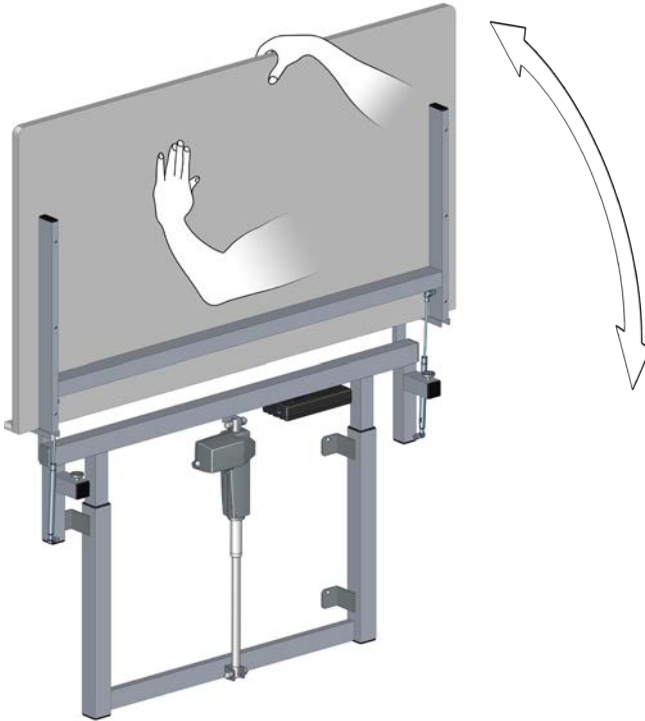
Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.

Never place objects underneath the changing table.

Folding up and down

The models R8721, R8921, R8722, R8922, R8723 can be folded up against the wall after use.

en

When folding the table up and down, make sure you support the lying surface using your hand and/or body to avoid risk of damage.

The built-in pneumatic gas cylinders make it easy to fold the table up and down.

Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

Maintenance

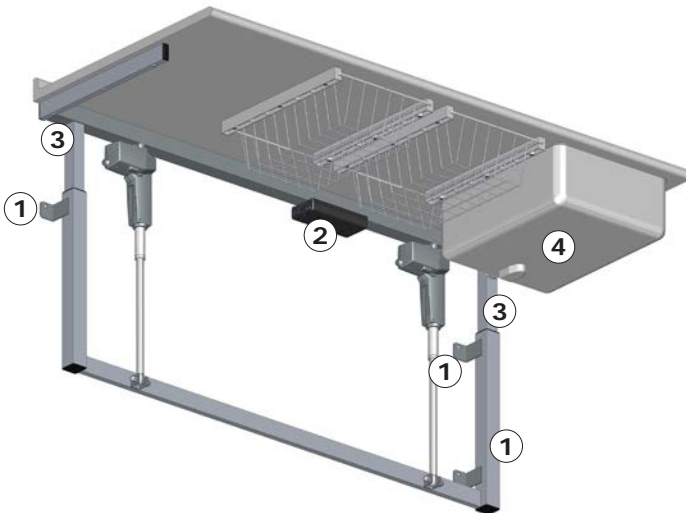
Expected service life of the changing table is 9 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Take the changing table to its highest position.

- Check the wall brackets ① and tighten the screws if necessary.
- Check all electrical connections to the control box ②.
- Oil moving parts, especially the sliding brackets ③.
- Check the sink drain ④ for leaks (if a sink is fitted).

Reconnect mains.

Product label

PRESSALIT®

1 **REF** RZZZZYYYYXXX-X

2 **SN** RZZZZYYYYXXX-X

3 YYYYY-MM-DD

4 12

5 13

6 XXX kg
XXX lbs

7 P:X W

8 U:XXXV

9 XX-XX Hz

10 IPXX

11 **Xxxxxx**

12 DS/EN XXXXX-X/X A1:YYYY

13 Pressalit A/S
Pressalitvej 1
DK-8680 Ry
+45 8788 8777
pressalit.com

14 **CE** **UK CA**

15 YYYYYYYY XXXXXX

16 **en**

① Article number

② Serial number

③ Production date

④ Consult manual for use

⑤ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑥ Product weight

⑦ Maximum power consumption

⑧ Electric current

⑨ Name and address of manufacturer

⑩ These products are CE-marked in accordance with relevant requirements in:

- Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

⑪ EAN 13 barcode

⑫ Consult the manual for important cautionary information such as warnings and precautions.

⑬ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑭ Duty cycle

⑮ Maximum user weight

⑯ Mains voltage

⑰ Frequency

⑱ Standard, which product is manufactured in accordance with (if any).

These products are UKCA-marked in accordance with relevant requirements in:

- Supply of Machinery (Safety) Regulations 2008.
- The restriction of the use of certain hazardous substances in electrical and electronic equipment regulations 2012.

A declaration of conformity can be found at pressalit.com.

Trouble-shooting

Symptom	Cause	Action
The product will not work, even though the wired remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".
The baskets are unable to close/open.	The basket may have been twisted unintentionally.	Try to straighten the basket.

Contact Pressalit A/S or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the warranty.

Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40° Celsius (41 and 104° F).
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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