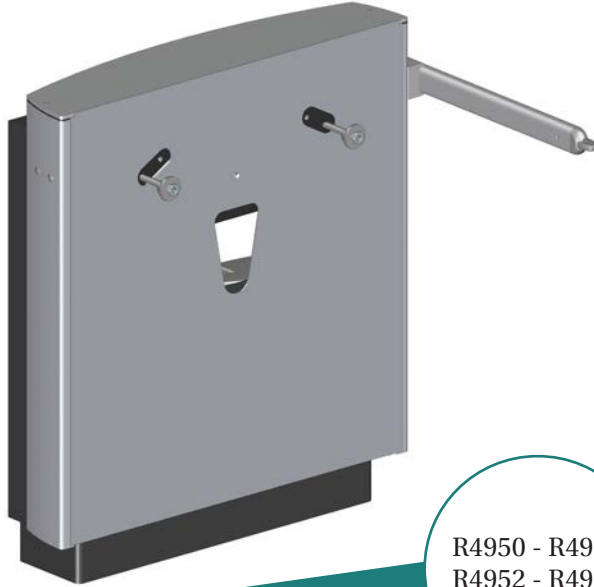
PRESSALIT®

R4950 - R4951  
R4952 - R4953

<b>SELECT wash basin lifter</b> – Operation and maintenance manual	<b>en</b>
<b>SELECT vaskeløfter</b> – Brugs- og vedligeholdelsesvejledning	<b>dk</b>
<b>SELECT Waschtisch-Lifter</b> – Gebrauchs- und Pflegeanleitung	<b>de</b>
<b>SELECT support de lavabo</b> – Manuel d'utilisation et d'entretien	<b>fr</b>
<b>SELECT muurframe voor wastafel</b> – Gebruiks- en onderhoudshandleiding	<b>nl</b>
<b>SELECT upphängning till tvättställ</b> – Bruks- och underhållsanvisning	<b>sv</b>
<b>SELECT oppheng til servant</b> – Bruks- og vedlikeholdsveiledning	<b>no</b>
<b>SELECT soporte de lavabo</b> - Instrucciones de uso y mantenimiento	<b>es</b>
<b>SELECT 洗面盆支架</b> - 操作和维护手册	<b>中文</b>

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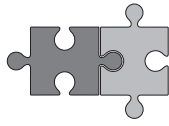
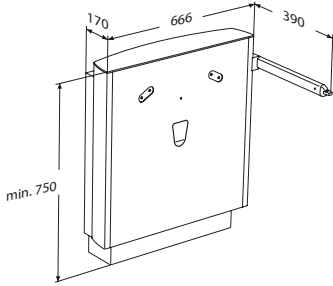
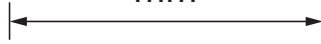
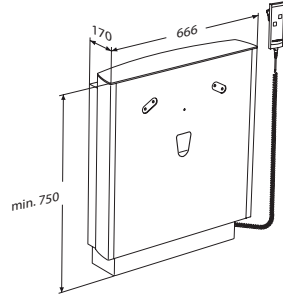
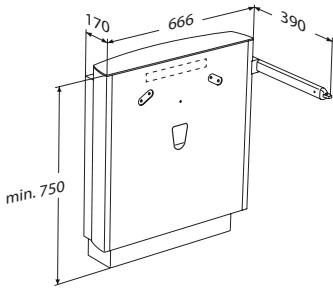
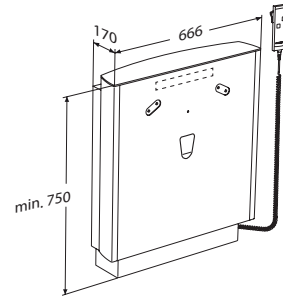
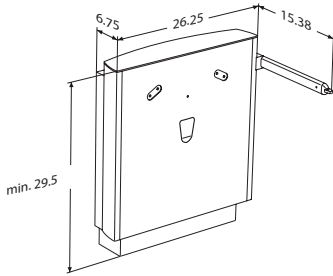
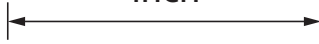
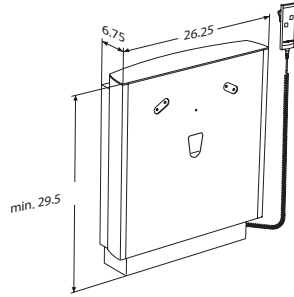
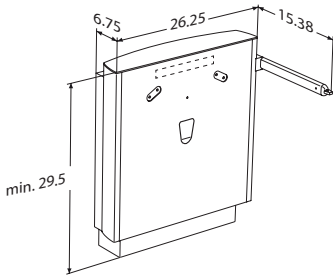
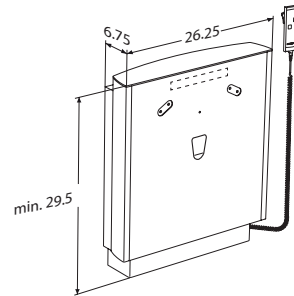
**R4953**

# Content

Mounting instruction .....	7
Symbols used in this manual .....	8
General safety .....	9
Product label .....	10
Intended use .....	12
Intended operator profile .....	12
Operating environment .....	12
Operating instructions .....	13
Safety stop function .....	14
To remove limescale .....	14
Maintenance .....	15
Service inspection .....	16
Technical data .....	17
Trouble-shooting .....	18
Disposal and recycling .....	19

## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [www.pressaliticare.com](http://www.pressaliticare.com).

Original instructions

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

Hand injury  
warning

Electricity or potentially  
dangerous voltage.

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

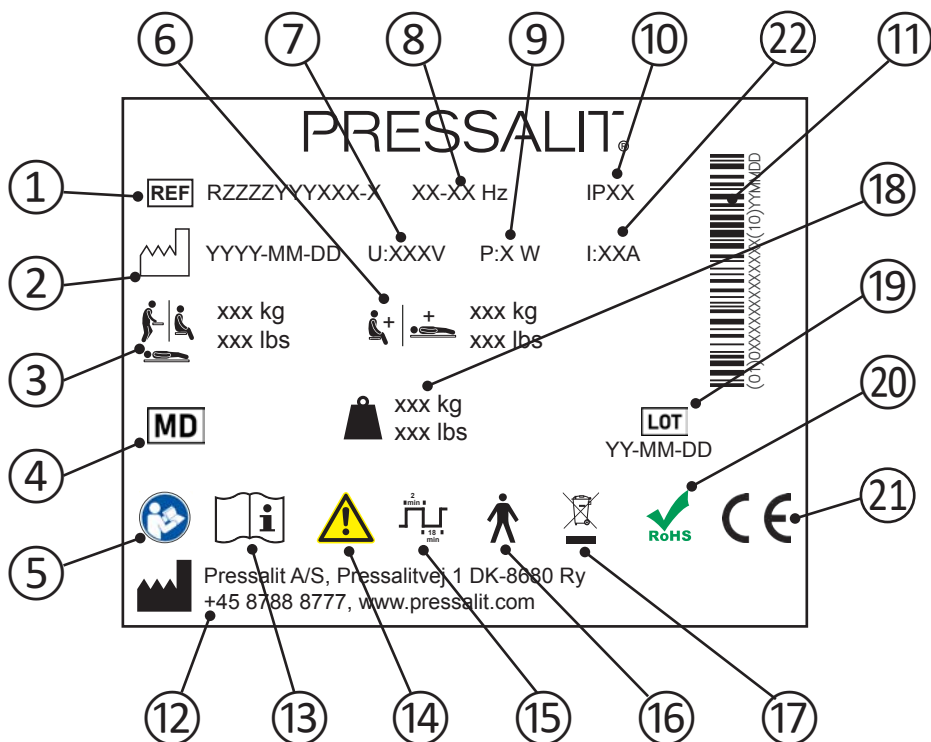
## **CAUTION**

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Product label

- ① Article number

---

- ② Production date

---

- ③ Maximum user weight

---

- ④ Medical device

---

- ⑤ Consult manual for use

---

- ⑥ Max load

---

- ⑦ Mains voltage

---

- ⑧ Frequency

---

- ⑨ Maximum power consumption

⑩ IP class

---

⑪ GS1-128 barcode

---

⑫ Name and address of manufacturer

---

⑬ Consult manual for use

---

⑭ Consult the manual for important cautionary information such as warnings and precautions

---

⑮ Duty cycle

---

⑯ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

---

⑰ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

---

⑱ Product weight

---

⑲ Batch code

---

⑳ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

---

These products are CE-marked in accordance with relevant requirements in:

- ㉑
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
  - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
  - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

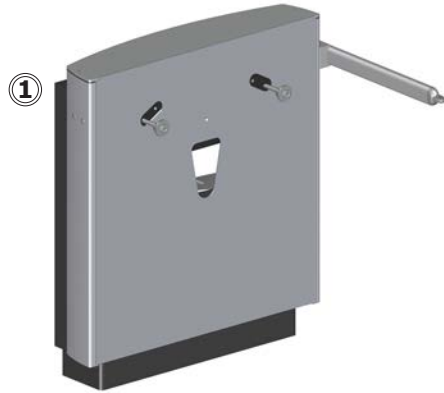
A declaration of conformity can be found at [www.pressalit.com](http://www.pressalit.com)

---

㉒ Electric Current

---

Product label placement ①.

## Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

## Intended operator profile

The product can be used by all - adults, children, young and old.

en

## Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in a stand-alone environment, in private homes, nursing homes and hospitals under normal working conditions.

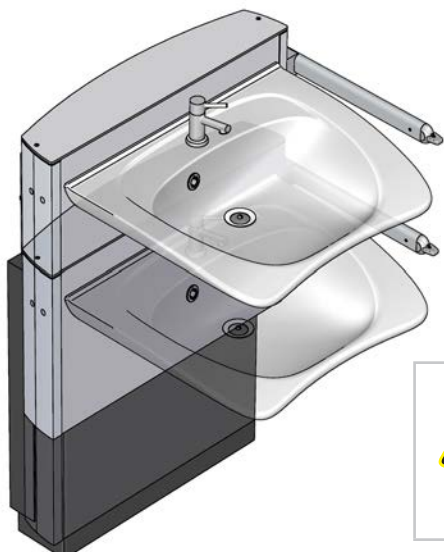
### **CAUTION**

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

### **NOTICE**

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.


# Operating instructions

The wash basin lifter can be adjusted 300 mm up and down.  
Max load is 150 kg.

**Take care to avoid injury to hands when in the vicinity of moving mechanical parts.**
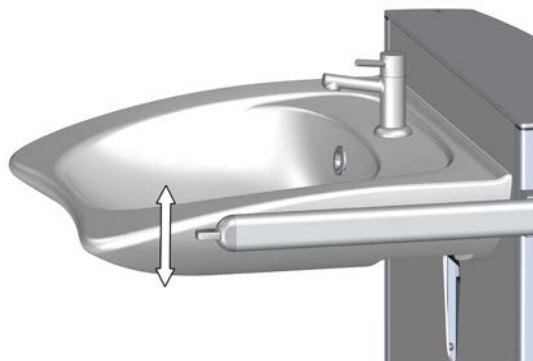
## **R4950 and R4952**

The height is adjusted by the lever control. Press and hold the lever up to raise the wash basin, and press and hold the lever down to lower the basin. When the lever is released, the motion stops.

en

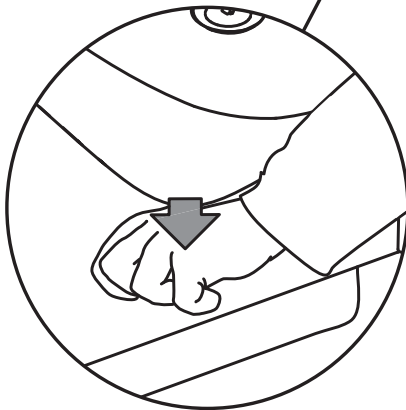
## **R4951 and R4953**

The height adjustment is controlled by the wired hand control unit.

Press and hold the Up button to raise the wash basin, and press and hold the Down button to lower it. When the button is released, the motion stops.

# Safety stop function

The wash basin lifters **R4952** and **R4953** are equipped with a safety stop function.

If the wash basin comes in contact with an object when it is being lowered, the motion stops immediately.

If an object comes in contact with the underside of the wash basin when it is stationary, the safety function is activated and the wash basin is lifted.

The wash basin lifter can be used again as soon as the obstruction has been removed.

**CAUTION**

Never place objects underneath the wash basin.

## To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts of 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the wash basin lifter is in use.

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on [www.pressalit.com](http://www.pressalit.com).

 **CAUTION**

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

**IMPORTANT**

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

## Maintenance

Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that service inspection is done once a year.

## Service inspection

Take the wash basin to its highest position.

**Disconnect mains before removing the covers.**

Remove the lower front cover ①. You do not need any tools, as the cover has a click lock.

Check all electrical connections for the control box ②.

Lubricate the inner square tubes ③. with WD-40 or equivalent. You have to reach up under the upper front cover to reach the tubes.

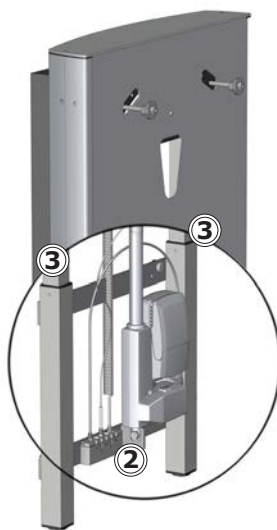
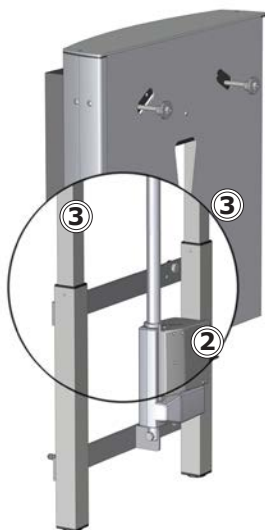
Reconnect mains.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the lower front cover.

**R4950 and R4951**

**R4952 and R4953**

# Technical data

Max load	150 kg
Materials	Powder-laquered stainless steel Aluminium Plastics
Dimensions (width x height x depth)	666 mm x 750 mm x 170 mm
Weight	R4950, R4952: 28.5 kg
	R4951, R4953: 28 kg
IP class	IPX6
<b>Power specifications</b>	
Max consumption	75 W
Voltage	Mains connection: 230V/120V/100V
Frequency	50/60 Hz
Current	3.7 A

# Trouble-shooting

Symptom	Cause	Action
The product will not work, even though the controls is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

<b>Recyclable components</b>	<b>Material</b>
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Polystyrene	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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