UK: Indivo lift units for the kitchen  
User and maintenance guide

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RK1010 – RK1011 – RK1012  
RK1013 – RK1014 – RK1020  
RK1021 – RK1081 – RK1082  
RK1083 – RK1100 – RK1101  
RK1110 – RK1111 – RK1112  
RK1113 – RK1114 – RK1181  
RK1182 – RK1183

English.....	3
Dansk .....	21
Norsk.....	39
Svenska .....	57
Deutsch .....	75
Français .....	93
Español.....	111
Nederlands .....	129

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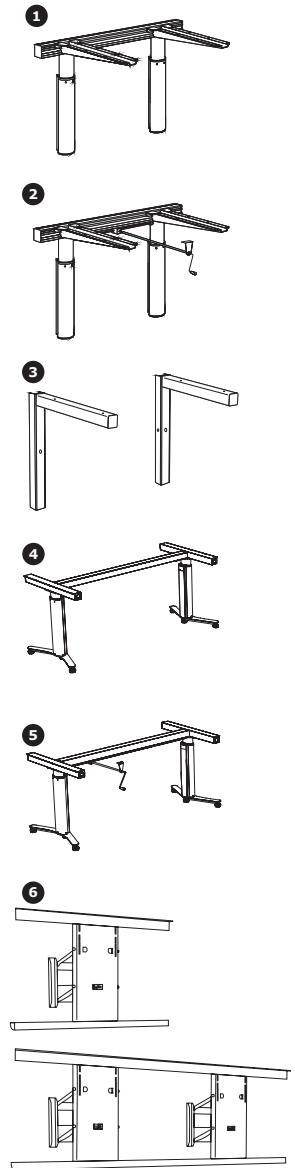
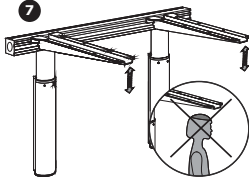
## Thank you for choosing this Pressalit Care lift unit for kitchens.

Combined with either a worktop or wall cabinet, the lift unit makes it possible to create a height-adjustable kitchen, ensuring the best working conditions for all users.

We hope that helpers and users will benefit from the multiple functions and advantages of the unit:

- Pressalit Care lift units are available in 6 variants:
  - ➊ Electrical lift units for worktops.
  - ➋ Manual lift units for worktops.
  - ➌ Manual lift units for worktops.
  - ➍ Electrical lift units for freestanding tables.
  - ➎ Manual lift units for freestanding tables.
  - ➏ Electrical lift units for wall cabinets.
- The flexibility of the units makes it easy for helpers and users to work together in the kitchen.
- The Indivo range includes three lift systems: electrical and manual - both of which can be adjusted daily - and infinitely adjustable, suitable for periodical adjustment.
- Indivo can be combined with worktops and wall cabinets at your discretion.
- Regardless of the combinations you choose, our safety systems provide optimum safety.

---

## Important safety information

Please read and become familiar with the user and maintenance guide before using the lift unit. Pressalit Care cannot be held liable if the product is used in any way which differs from that described in the guide.

- The information in the guide is based on correct fitting of the lift unit in accordance with our installation guide.
- If the product is defective or damaged, it must not be used before it has been repaired by a service engineer.
- For safety reasons, nothing should obstruct the lift unit when in movement. 7
- The product is not intended for play, and we therefore recommend that it is not operated by children without adult supervision.
- We reserve the right to amend this guide without prior notice.

---

## Before using the lift unit for the first time

- Users, helpers and therapists must read and be familiar with this user and maintenance guide before using the unit.
- Keep this guide in the immediate proximity of the lift unit.
- Check that the unit's control box (models with electric motor only) is suitable for the mains electricity supply.
- Kitchen lift units are for indoor use only.

## Electrical lift units

### Maximum load

#### Wall-mounted 8

100 kg, incl. worktop.

#### Freestanding 9

100 kg, incl. worktop.

#### Wall cabinet 10

1 motor bracket: 110 kg incl. cabinet

2 motor brackets: 220 kg incl. cabinet

### Operation

- The height of the lift units can be adjusted using the switches on the frame.
- As an alternative or supplement to the frame-mounted switches 11 a hand-held control unit is available. 13

### Adjustment

#### Wall-mounted

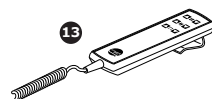
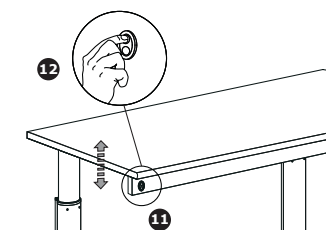
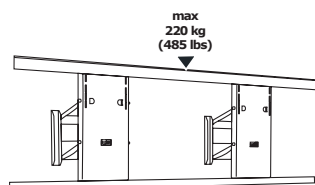
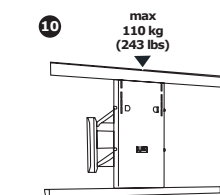
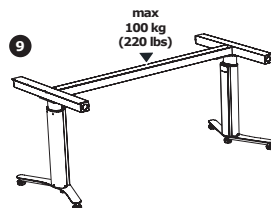
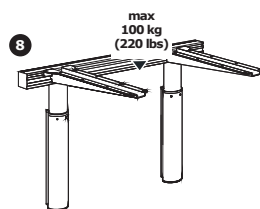
- The height of the electric lift unit can be set using the frame-mounted switch 11 or hand-held remote control 13, which has an up/down indicator. Raise the unit by pressing and holding the upper part of the indicator. 12 Lower the unit by pressing and holding the lower part of the indicator. Release the indicator to stop the unit.
- When the unit is mounted at the recommended height, it can be adjusted between 670 and 950 mm above the floor.

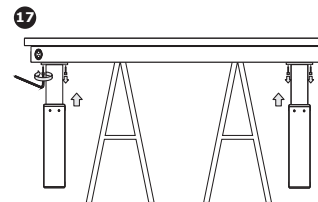
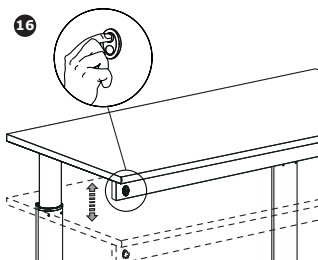
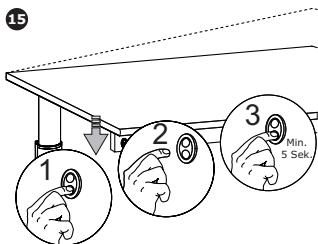
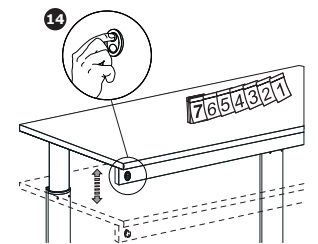
#### Freestanding

- The height of the electric lift unit can be set using the frame-mounted switch 11, which has an up/down indicator. Raise the unit by pressing and holding the upper part of the indicator. 12 Lower the unit by pressing and holding the lower part of the indicator. Release the indicator to stop the unit.
- The unit can be adjusted to heights of between 680 and 980 mm above the floor.

#### Wall cabinet

- The height of the lift unit can be adjusted using the switches on the frame 11 or hand-held remote control. 13
- When wall cabinets are lowered 430 mm to their lowest position, they are also projected 180 mm towards the user.

## Maintenance

- We recommend running the motors on the unit to their maximum positions at least once a week by raising the unit to its full height, and then lowering it to its lowest position, so that the internal, self-lubricating parts are lubricated.<sup>14</sup>
- Reset the unit to maintain its smooth up/down movement.<sup>15</sup> Press the indicator until the unit stops at its lowest position - release the lower indicator - press once more and hold for 5 seconds.

## Service inspection

We recommend that a service inspection is performed once annually.

### Wall-mounted

1. Run the unit up/down <sup>16</sup> to check that:
  - a. there are no visible defects or abnormal noise.
  - b. the product runs up/down satisfactorily.

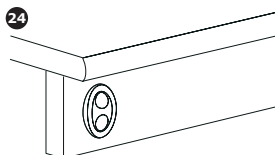
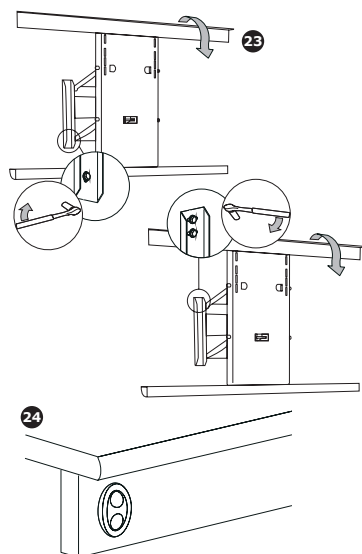
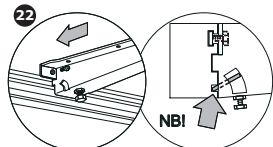
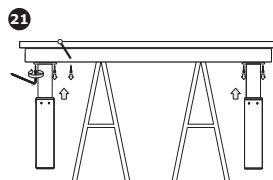
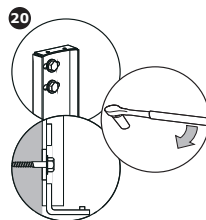
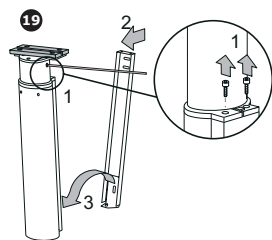
If there are no visible defects or abnormal noises coming from the moving parts, no further action is required. If a telescopic leg is defective, it must be replaced. (Contact your dealer for replacement).
2. Check the wall brackets and tighten if necessary. If the brackets need tightening, raise the worktop to maximum height. Support and remove the worktop with its longitudinal profile.<sup>17</sup> Lower the telescopic legs to their lowest position.<sup>18</sup> Slacken the two screws behind the legs, and remove the legs.<sup>19</sup> Tighten wall bracket bolts.<sup>20</sup> Replace the telescopic legs in reverse order, and raise them. Fit the worktop with its longitudinal profile <sup>21</sup>.
3. Check all electrical connections to the control box. If the transformer is defective, it must be replaced (contact your dealer for replacement).
4. Check the supports are correctly fitted.<sup>22</sup>
5. Check the operating switch.<sup>23</sup> If the switch is defective, it must be replaced. (Contact your dealer for replacement).
6. Check for broken cables or wires.
7. Check the safety device (if fitted in the kitchen). Check that the safety device rails/plates can move freely and return to their neutral position. Check whether the lift unit stops when the safety device is operated.

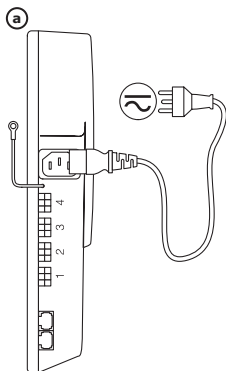
### Freestanding

1. Run the lift unit up/down to check that:
  - a. there are no visual defects or abnormal noise.
  - b. the product runs up/down satisfactorily.
 If there are no visible defects or abnormal noises coming from the moving parts, no further action is required.  
 If a telescopic leg is defective, it must be replaced. (Contact your dealer for replacement).
2. Check all electrical connections to the control box. If the transformer is defective, it must be replaced. (Contact your dealer for replacement).
3. Check the operating switch. <sup>23</sup> If the switch is defective, it must be replaced. (Contact your dealer for replacement).

### Wall cabinet

1. Run the lift unit up/down to check that:
  - a. there are no visual defects or abnormal noise.
  - b. the product runs up/down satisfactorily.
 If there are no visible defects or abnormal noises coming from the moving parts, no further action is required.  
 If the unit is defective, it must be replaced. (Contact your dealer for replacement).
2. Check the wall brackets and tighten if necessary. If the wall units have to be tightened, run the unit out/down a little, and tighten the lower screws. Run the unit all the way out/down, and tighten the upper screws. <sup>23</sup>
3. Check all electrical connections to the control box. If the transformer is defective, it must be replaced (contact your dealer for replacement).
4. Check the operating switch. <sup>24</sup> If the switch is defective, it must be replaced. (Contact your dealer for replacement).


## Faultfinding


### Wall-mounted lift unit for worktop

Fault	Cause	Remedy
Lift unit for worktop <u>without safety rail</u> cannot be height-adjusted.	No power.	Check mains electricity is correctly connected to the control box. <b>a</b> (The control box will usually be located under the worktop).
	Telescopic legs not mounted at the same height.	Mount the telescopic legs at the same height and level.
	A sink or installation cover collides with an obstruction.	Check whether the installation cover comes into contact with the skirting board or the like.
	Lift unit has not been reset (e.g. after power loss).	Reset the unit if possible: 1. Activate the indicator until the unit is in its lowest position. Release the switch. 2. Activate the lower indicator again for 5 seconds. 3. The lift unit is now reset. (It may be necessary to reset the unit several times in succession).
Faulty plugs.	Check plug from motor to extension lead, and those in inlets 1 and 2 on the control box. The male plugs have small steel pins which can be easily bent when inserted into the control box. If one of the pins is bent, straighten it out before replacing the plug in the control box. REMEMBER to reset.	
The frame-mounted switch cannot be operated easily.	Check whether the switch can be operated freely and that it returns automatically to neutral position. The movement can be hindered by dirt or grease.  Check the frame-mounted switch: Check that all wires are correctly connected according to the installation guide and that none are broken.  If a wire is incorrectly connected, the lift unit cannot run up or down.  NB: If the lift unit will not rise, the switch can be considered as defective.  If all wires from the switch are correctly connected and none are broken, the lift unit should be able to run up/down. REMEMBER to reset.	

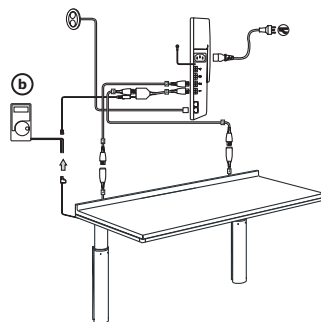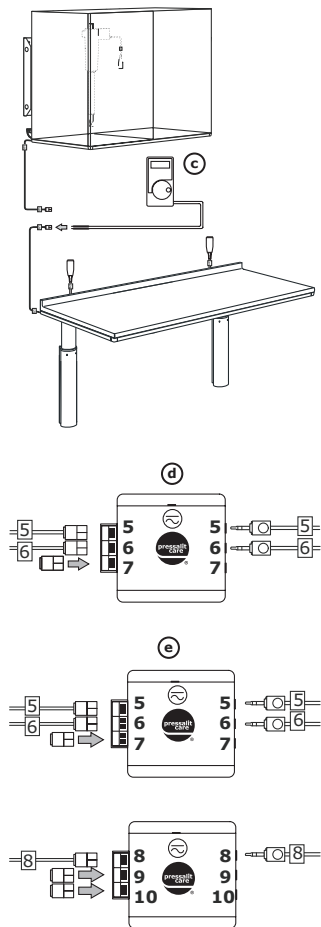Lift unit for worktop <u>with safety rail</u> cannot be height-adjusted.	Fault in safety device.	The worktop cannot run up or down if the safety rail is defective. Check that all wires and plugs are intact and correctly connected according to the installation guide.
	Installation error on safety rail.	Use measurement device <b>(b)</b> (ohmmeter) to detect current. Measure current on the wire from the safety rail and its plug (a close function). - if there is current, the safety device system is working. The lift unit should now run up/down. REMEMBER to reset. - if no current can be detected, remove the plastic covers and slacken the screws at the end of the safety rail. The rail should now move freely. Note that the rail's freedom to move can be hindered by dirt or grease.  - check for current once more: If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".
	Faulty switches.	Check whether one or more of the safety system contacts is sticking, and if so, attempt repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. REMEMBER to reset.



<p>Lift unit for worktop and wall cabinet with <u>safety rail/plate</u>, cannot be height-adjusted</p>	<p>Fault in safety system.</p>	<p>The worktop/wall cabinet cannot run up or down if the safety rail/plate is defective. Check that all wires and plugs are intact and correctly connected according to the installation guide.</p>
	<p>Installation error on safety rail/plate.</p>	<p>Use measurement device <b>(c)</b> (ohmmeter) to detect current. Measure current on the wires from the safety rail/plate and their plug (a close function).          - if there is current, the safety device system is working. The lift units should now run up/down. REMEMBER to reset.          - if no current can be detected:          a. remove the plastic corners on the safety rail, and slacken the screws at both ends. The rail should now move freely. Note that the rail's freedom to move can be hindered by dirt or grease.          Check for current once more:          If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".          b. slacken all screws on the safety plate until it can move freely. Note that the plate's freedom to move can be hindered by dirt or grease.          Check for current once more:          If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".</p>
	<p>Faulty switches.</p>	<p>Check whether one or more of the safety system contacts is sticking, and if so, attempt repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. REMEMBER to reset.</p>
	<p>Jumper missing in safety device box.</p>	<p>If there is a combination in which one of the safety inputs in the safety device box is not in use, that input must be fitted with a jumper lead. If a jumper lead is missing, the worktop will not run, although the wall cabinets will run up and down correctly. See installation guide.          When jumper lead and other safety device plugs <b>(d)</b> (mk. 5 and 6) are inserted, both worktop and wall cabinet will run up and down correctly.          Note that in some instances, 2 safety device boxes are used <b>(e)</b>, of which one is marked 8-9-10.          If the lift units run up/down correctly, all safety systems are working correctly. REMEMBER to reset.</p>

## Freestanding lift units for worktop

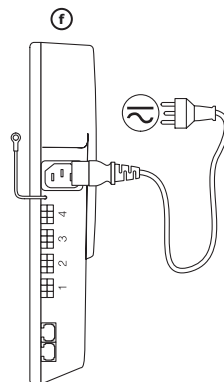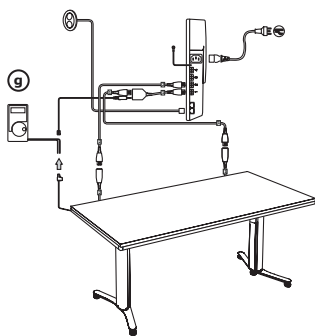Fault	Cause	Remedy
Lift unit for worktop <u>without safety rail</u> cannot be height-adjusted	No power.	Check mains electricity is correctly connected to the control box.  (The control box will usually be located under the worktop).
	Lift unit has not been reset (e.g. after power loss).	Reset the unit if possible: 1. Activate the indicator until the unit is in its lowest position. Release the switch. 2. Activate the lower indicator again for 5 seconds. 3. The lift unit is now reset. (It may be necessary to reset the unit several times in succession).
	Faulty plugs.	Check plug from motor to extension lead, and those in inlets 1 and 2 on the control box. The male plugs have small steel pins which can be easily bent when inserted into the control box. If one of the pins is bent, straighten it out before replacing the plug in the control box. REMEMBER to reset.
	The frame-mounted switch cannot be operated easily.	Check whether the switch can be operated freely and that it returns automatically to neutral position. The movement can be hindered by dirt or grease. Check the frame-mounted switch: Check that all wires are correctly connected according to the installation guide and that none are broken. If a wire is incorrectly connected, the lift unit cannot run up or down. NB: If the lift unit will not rise, the switch can be considered as defective. If all wires from the switch are correctly connected and none are broken, the lift unit should be able to run up/down. REMEMBER to reset.



<p>Lift unit for worktop <u>with</u> safety rail cannot be height-adjusted.</p>	<p>Fault in safety system.</p>	<p>The worktop cannot run up or down if the safety rail is defective. Check that all wires and plugs are intact and correctly connected according to the installation guide.</p>
	<p>Assembly error on safety rail system.</p>	<p>Use measurement device ⑨ (ohmmeter) to detect current. Measure current on the wire from the safety rail and its plug (a dose function). - if there is current, the safety device system is working. The lift unit should now run up/down. REMEMBER to reset. - if no current can be detected: Remove the plastic corners on the safety rails and slacken the screws at both ends. The rails should now move freely. Note that the rails' freedom to move can be hindered by dirt or grease. Check for current once more: If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".</p>
	<p>Faulty switches.</p>	<p>Check whether one or more of the safety system contacts is sticking, and if so, attempt repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. REMEMBER to reset.</p>

## Lift units for wall cupboards

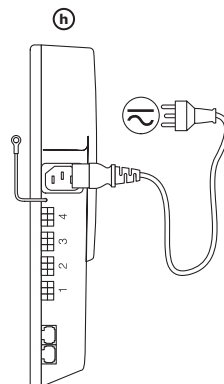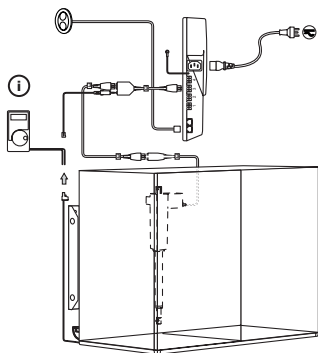Fault	Cause	Remedy
Lift unit for wall cupboard <u>without safety plate</u> cannot be height-adjusted.	No power.	Check mains electricity is correctly connected to the control box. <b>(h)</b> (The control box will usually be located under the worktop).
	Lift unit has not been reset (e.g. after power loss).	Reset the unit if possible: 1. Activate the indicator until the unit is in its lowest position. Release the switch. 2. Activate the lower indicator again for 5 seconds. 3. The lift unit is now reset. (It may be necessary to reset the unit several times in succession).
	Faulty plugs.	Check plug from motor to extension lead, and those in inlets 3 and 4 on the control box. The male plugs have small steel pins which can be easily bent when inserted into the control box. If one of the pins is bent, straighten it out before replacing the plug in the control box. <b>REMEMBER</b> to reset.
	The frame-mounted switch cannot be operated easily.	Check whether the frame-mounted switch can be operated freely and that it returns automatically to neutral position. The movement can be hindered by dirt or grease. Check the frame-mounted switch: Check that all wires are correctly connected according to the installation guide and that none are broken. If a wire is incorrectly connected, the lift unit cannot run up or down. NB: If the lift unit will not rise, the switch can be considered as defective. If all wires from the switch are correctly connected and none are broken, the lift unit should be able to run up/down. <b>REMEMBER</b> to reset.



Lift unit for wall cupboard with safety plate cannot be height-adjusted

Fault in safety system.	The wall cabinet cannot run up or down if the safety plate is defective. Check that all wires and plugs are intact and correctly connected according to the installation guide.
Assembly error on safety plate.	<p>Use measurement device ① (ohmmeter) to detect current. Measure current on the wire from the safety rail and its plug (a close function).</p> <ul style="list-style-type: none"> <li>- if there is current, the safety device system is working. The lift unit should now run out/down. REMEMBER to reset.</li> <li>- if no current can be detected: Slacken the screws on the safety plate. The plate should now move freely. Note that the plate's freedom to move can be hindered by dirt or grease.</li> </ul> <p>Check for current once more: If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".</p>
Faulty switches.	Check whether one or more of the safety system contacts is sticking, and if so, attempt repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. REMEMBER to reset.

Lift units for worktop and wall cabinet with safety plates/rails cannot be height-adjusted.	Fault in safety system.	The worktop/wall cabinets cannot run up or down if the safety rail/plate is defective. Check that all wires and plugs are intact and correctly connected according to the installation guide.
	Installation error on safety rail/plate.	<p>With measurement device (ohmmeter)  to detect current. Measure current on the wires from the safety rail/plate and their plug (a close function).</p> <ul style="list-style-type: none"> <li>- if there is current, the safety device system is working. The lift units should now run up/down. REMEMBER to reset.</li> <li>- if no current can be detected: <ul style="list-style-type: none"> <li>a. slacken all screws on the safety plate until it can move freely. Note that the plate's freedom to move can be hindered by dirt or grease.</li> </ul> </li> </ul> <p>Check for current once more: If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".</p> <ul style="list-style-type: none"> <li>b. remove the plastic corners on the safety rail, and slacken the screws at both ends. The rail should now move freely. Note that the rail's freedom to move can be hindered by dirt or grease.</li> </ul> <p>Check for current once more: If there is current, the safety device system is working. If no current can be detected, see under "Faulty switches".</p>
	Faulty switches.	Check whether one or more of the safety system contacts is sticking, and if so, attempt repair. If the problem cannot be fixed, replace the contact(s). The safety system will only work when current can be detected. REMEMBER to reset.
	Jumper lead missing in fuse box.	<p>If there is a combination in which one of the safety inputs in the safety device box is not in use, that input must be fitted with a jumper lead. If a jumper lead is missing, the worktop will not run, although the wall cabinets will run up and down correctly. See installation guide.</p> <p>When jumper lead and other safety device plugs (mk. 5 and 6) are inserted, both worktop and wall cabinet will run up and down correctly.</p> <p>Note that in some instance, 2 safety device boxes are used, of which one is marked 8-9-10.</p> <p>If the lift units run up/down correctly, all safety systems are working correctly. REMEMBER to reset.</p>

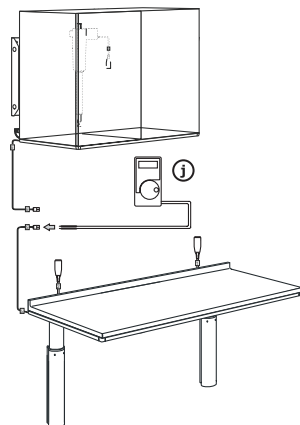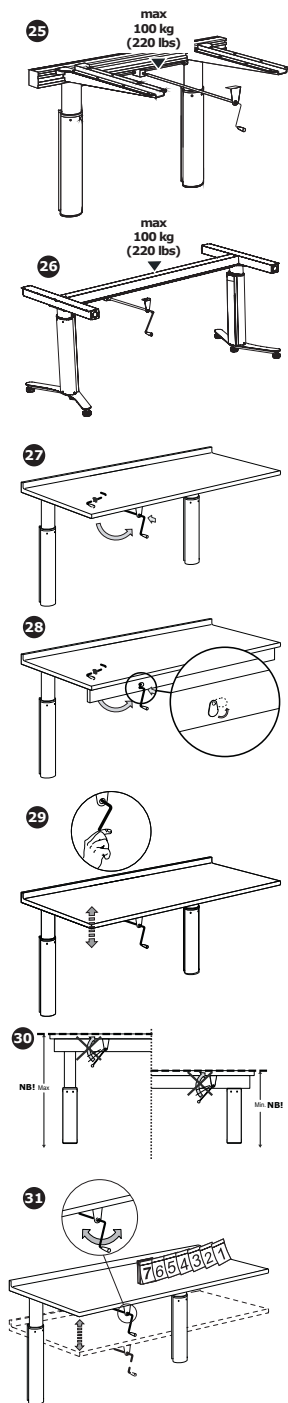Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

## Manual lift units

### Maximum load

**Wall-mounted** 25  
100 kg, incl. worktop.

**Freestanding** 26  
100 kg, incl. worktop.

### Operation

#### Wall-hanging/freestanding

Remove 27 hand crank from under worktop and fit onto drive shaft. If there is a frame around the worktop, push the 28 over the drive shaft to one side and fit the hand crank.

### Adjustment

#### Wall-hanging/freestanding

- Adjust the manual lift unit for height by turning the hand crank. 29
- Always stop turning the crank when you feel resistance at either the upper or lower position. 30
- The unit can be adjusted to heights of between 680 and 980 mm above the floor.

### Maintenance

- We recommend winding the spindle on the unit to the maximum positions at least once a week, by raising the unit to its full height, and then lowering it to its lowest position, so that the internal, self-lubricating parts are lubricated. 31

## Service inspection

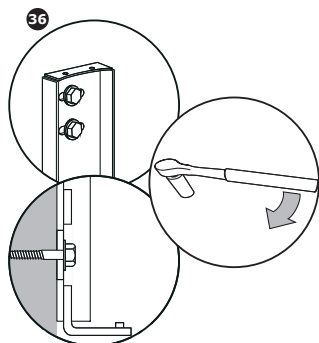
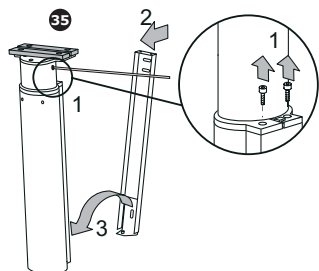
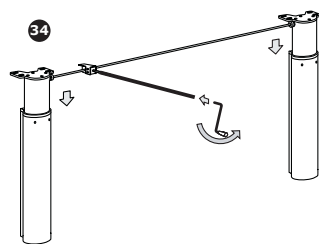
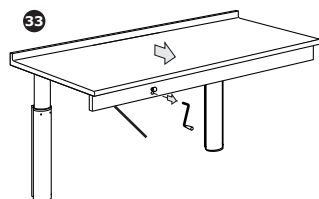
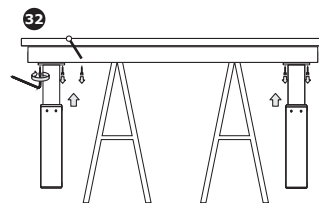
We recommend that a service inspection is performed once annually.

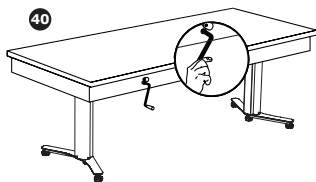
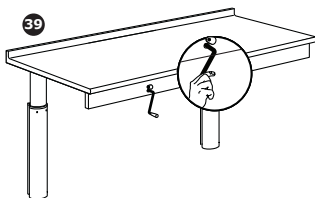
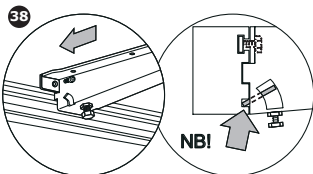
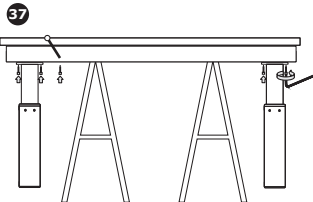
### Wall-mounted

- Run the lift unit up/down to check that:
  - there are no visual defects or abnormal noise.
  - the product runs up/down satisfactorily.
 If there are no visible defects or abnormal noises coming from the moving parts, no further action is required. If a telescopic leg is defective, it must be replaced. (Contact your dealer for replacement).
- Check the wall brackets and tighten if necessary. If the brackets need tightening, raise the worktop to maximum height. Support the worktop with its longitudinal profile, and remove the angle gear and profile.<sup>32</sup> Remove the worktop. The crank extension rod will hang free.<sup>33</sup> Insert the crank in the extension rod, wind the worktop down and remove.<sup>34</sup> Slacken the two screws behind the legs, and remove the legs.<sup>35</sup> Tighten wall bracket bolts.<sup>36</sup> Replace the telescopic legs in reverse order, and raise them. Fit the worktop with its longitudinal profile<sup>37</sup>
- Check the supports are correctly fitted.<sup>38</sup>
- Check the crank.<sup>39</sup> If the crank is defective, it must be replaced. (Contact your dealer for replacement).

### Freestanding

- Run the lift unit up/down to check that:
  - there are no visual defects or abnormal noise.
  - the product runs up/down satisfactorily.
 If there are no visible defects or abnormal noises coming from the moving parts, no further action is required. If a telescopic leg is defective, it must be replaced. (Contact your dealer for replacement).
- Check the legs and tighten if necessary.
- Check the crank.<sup>40</sup> If the crank is defective, it must be replaced. (Contact your dealer for replacement).

## Faultfinding

### Wall-mounted

Fault	Cause	Remedy
Lift unit does not run synchronously.	The telescopic legs are not mounted at the same height (not level).	Run the unit as far down to its lowest position as possible. Detach the longitudinal shaft on the side where the leg is highest. Insert an Allen key into the leg and turn until the leg is fully retracted.
The lift unit does not move when the hand crank is turned.	The crank is not correctly fitted on the drive shaft.	Push the crank down as far as it will go and turn until the worktop moves up and down.
	The extension rod is not correctly attached to the angle gear.	Connect the rod correctly to the gear. (See installation guide).
	The angle gear is not correctly attached to the longitudinal drive shaft between the two telescopic legs.	Connect the gear correctly to the drive shaft. (See installation guide).

### Freestanding

Fault	Cause	Remedy
Lift unit does not run synchronously.	The telescopic legs are not mounted at the same height (not level).	Run the unit as far down to its lowest position as possible. Detach the longitudinal shaft on the side where the leg is highest. Insert an Allen key into the leg and turn until the leg is fully retracted.
The lift unit does not move when the hand crank is turned.	The crank is not correctly fitted on the drive shaft.	Push the crank down as far as it will go and turn until the worktop moves up and down.
	The extension rod is not correctly attached to the angle gear.	Connect the rod correctly to the gear. (See installation guide).
	The angle gear is not correctly attached to the longitudinal drive shaft between the two telescopic legs.	Connect the gear correctly to the drive shaft. (See installation guide).

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

## Infinitely variable lift units

### Maximum load 41

75 kg, incl. worktop.

### Operation

When the infinitely variable lift unit is fitted in accordance with the installation guide, the unit is ready for use.

### Adjustment

Adjust the unit by slackening the screw. The unit can then be set to the desired height. Retighten the screw. 42

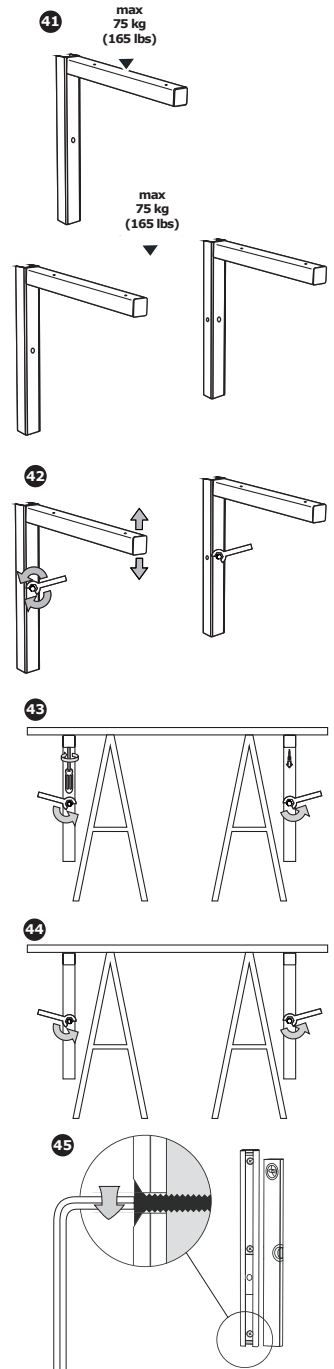
### Service inspection

We recommend that a service inspection is performed once annually.

1. Check the supports. Replace any broken supports. Support the worktop and remove it from the support. 43 Detach the support from the wall bracket by unscrewing and removing the bolt. 43 Fit a new support in the reverse sequence.
2. Check the wall brackets and tighten if necessary. Remove supports when tightening (including worktop) from the wall bracket by unscrewing and removing the bolts. 44 The wall brackets can now be tightened. 45 Replace the parts in reverse sequence.

### Faultfinding

Fault	Cause	Remedy
Worktop not level.	The telescopic legs are not mounted at the same height (not level).	If the supports cannot be height adjusted, check the threaded insert mounted inside the bracket. There must be no screw heads in the way of the threaded inset.  If the supports cannot be secured at a given height, we recommend adjusting the height a little before trying again.

---

## **General**

---

### **Cleaning**

#### **Daily**

Use ordinary soapy water. Avoid use of abrasive, corrosive or chlorine-based cleaning products. Rinse off with clean water and dry with a soft cloth.

These products cannot be cleaned in an autoclave.

#### **Occasional**

Use 3 parts 30% household vinegar to 7 parts water for removal of lime deposits. Wipe off with a cloth dipped in clean water and wrung out.

---

### **CE**

The CE mark indicates that this product has been manufactured according to the Machine Directive 2006/42/EC of 17 May 2006.

---

### **Information on disposal and recycling**

Electrical and electronic equipment (EEE) contains materials, components and substances which can be hazardous to human health and the environment when waste from electric and electronic equipment (WEEE) is not disposed of correctly. Such equipment must not be disposed of with unsorted domestic refuse, but must be collected separately. Waste must be processed according to local authority instructions.

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**Pressalit A/S**