

PRESSALIT.

Product	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9"
Item number	RT138108
	VVS:777568708
	NRF:6317191
	EAN:5708590154077
	HMI: 25429
	NBS:N13:436
Color	Dark blue (RAL 5011)
Product description	PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroor A reliable solution with many different options, which can easily lay the grounds for increased independence.
	PLUS grab bar combination, 1.3", reversible right/left, A:30" x B:17.9" x C: 42.9". Material: Powder-coated aluminium and plastic. Excl. mounting kit.
Product capacity	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.
Materials	Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic Strap: Nylon (PA)
Ergonomy	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection 2" from the wall makes it easy to take hold of the rail.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT.

Product Item number	PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9" RT138108
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
De de est	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.
Declaration of conformity	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.



PRESSALIT.

Prod	luct
Item	number

PLUS grab bar, corner with vertical grip 30" x 17.9" x 42.9"

RT138108


Mounting in different wall types

\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\	, , , , , ,
2 2 . 2	. 7. 4
0.000	? c · · · ·

Concrete >B25

V8618 - Mounting kit, 16 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.




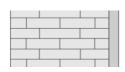

Solid Brick >= Mz12

V8618 - Mounting kit, 16 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.



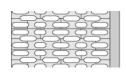

Solid Sand Lime Brick >= Hlz12


V8618 - Mounting kit, 16 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






Aerated Concrete P4

V8668 - Mounting kit, 16 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.

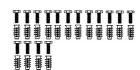

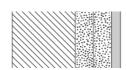

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8618 - Mounting kit, 16 bolts \emptyset 5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.





PRESSALIT®

12,5 mm) water-resistant, re-enforced with ood rs (16 pcs) must be bought separately, for ossible to use e.g. SnapTight or WingIt milar.
12,5 mm) water-resistant, re-enforced with (.08 inch) rs (16 pcs) must be bought separately, for ossible to use e.g. SnapTight or WingIt milar.
12,5 mm) water-resistant his wall type is not recommended - risk of persons and walls.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com