

PRESSALIT®

	PLUS wash basin bracket with wired hand control, electrically height
Product	adjustable and manually sideways adjustable
Item number	R4781
	VVS:654003220
	NRF:6132538
	RSK:7972596
	EAN: 5708590309347
	HMI: 71755
	NBS:N13:30,335
Product description	PLUS wash basin brackets are available in electrically, pneumatically and manually height adjustable versions. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.
	PLUS wash basin bracket, powered with wired remote control, electrically height adjustable by 300 mm and sideways adjustable. Recommendation: Flexible feed and waste system R2064/R2065/R2067. (UK markets R2074/R2075/R2077). Materials: Aluminium and ABS polymer. Incl. fixing bolts for wash basin. For fitting to horizontal PLUS wall track (not included).
Adjustability	Height and sideways adjustable
Adjustment height	300 mm
Product capacity	Maximum load: 100 kg. Suitable for wash basins weighing no more than 22 kg. When the wash basin is loaded with 100 kg, each wall fastener will be carrying 22 kg. Wash basin brackets need to be able to carry the weight of the wash basin, as well as support the weight of the person who is leaning against the wash basin or who is using the wash basin to rise from seated position. Neither wash basin nor wash basin bracket are intended to carry the full body weight of a person, and the test methods applicable to wash basin brackets reflect this: ISO 17966 prescribes a static test on the front edge of the wash basin at 75% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 100 kg and the wash basin weighs 22 kg, it is – according to ISO 17966 – the equivalent of a user weighing 104 kg leaning on the front edge of the wash basin.
Speed upwards	Speed up mm: 20 mm/s Speed down mm: 23 mm/s



PRESSALIT.

Product	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable
Item number	R4781
Noise level	45 dB
Materials	The wash basin bracket is primarily made of 10-micrometre anodized aluminium. The ends of the bracket cover are capped with ABS polymer. The brackets are available with a wired hand control encased in polypropylene.
Ergonomy	The height of the wash basin is easily adjusted by means of the wired hand control. The wash basin bracket suits all types of bolt-mounted wash basin. Its rounded, slee design greatly facilitates cleaning.
Features	An independant user will be able to make the necessary adjustments to an electrically operated wash basin bracket themselves without any problems.
Mains connection (V)	230
Actuator (motor)	LA40 (Linak) 4000N, IPX6
Control box	SMPS14002
Wired remote control	HB3 (Linak) IPX6
Power consumption (maximum load)	Power consumption by maximum load, watt: 75 W Power consumption by standby: 1,5 W
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

Product Item number	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable R4781
	Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S
	Erisan OXY+ Sprit (husholdningssprit)
	Suma Bac D10 Virkon
	Rodalon Desinfektionsspray Lysoformin Spezial
	CaviCide
	Sufanios Premium Tvättsprit 70%
	NON-DISINFECTANTS: Suma Star free D1
	Taski 100
	Taski 200 Milizid Citro
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other
	aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.
Declaration of conformity	CE UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of:
	- Regulation (EU) 2017/745 on medical devices
	 Council Directive 2006/42/EC on machinery Council Directive 2011/65/EC on RoHS. For the UK:
	- Medical Device Regulation (MDR) 2002 on medical devices

- Medical Device Regulation (MDR) 2002 on medical devices.

Equipment Regulations 2012.

Supply of Machinery (Safety) Regulations 2008
Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic



PRESSALIT®

Product	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable
Item number	R4781

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com