




|  |  |
|--|--|
| <p><b>Product</b><br/><b>Item number</b></p>             | <p>PLUS backrest, fixed height, for toilet lifter<br/>R6408112</p>   |
| <p><b>Colour</b><br/><br/><b>Product description</b></p> | <p>VVS:777530312<br/>NRF:6317037<br/>EAN:5708590365428<br/>HMI: 124218</p> <p>Anthracite grey</p>  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel.<br/>Fixed height, for mounting directly on SELECT TL1, TL2, TL3 or TL4 toilet lifter.<br/>Materials: Aluminium and polyurethane foam with a plywood core.<br/>Mounting kit (for mounting on toilet lifter) included.</p> |
| <p><b>Adjustability</b><br/><b>Product capacity</b></p>  | <p>Fixed mounting</p> <p>Maximum load: 50 kg.<br/>Maximum user weight: 100 kg.<br/>When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg.<br/><i>Backrests need to be able to support the weight of the person who is leaning against it.</i><br/><b><i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i></b></p>                                     |
| <p><b>Materials</b><br/><b>Ergonomy</b></p>              | <p>The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.</p> <p>Soft and slip-resistant material that has a warm feel.</p>   |
| <p><b>Warranty</b></p>                                   | <p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>   |

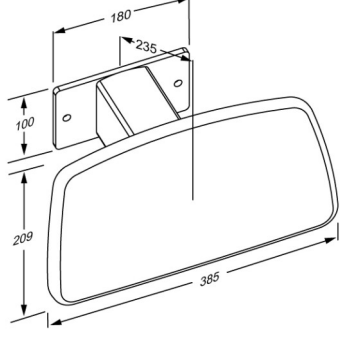

|                                  |  |
|----------------------------------|--|
| <b>Product</b>                   | PLUS backrest, fixed height, for toilet lifter   |
| <b>Item number</b>               | R6408112   |
| <b>Cleaning</b>                  | <p>PLUS backrest.<br/>The product is to be cleaned with hot water or a mild soapy cleaning agent.<br/>Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Erisan OXY+<br/>Tvättsprit 70%<br/>CaviCide surface disinfectant decontaminant cleaner<br/>Rodalon Desinfektionsspray<br/>Terralin protect (SCHÜLKE+)<br/>Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b><br/>Wetrok Caledor (Wetrok)<br/>Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> <p>CE UK MDR This product is in compliance with all relevant requirements of:<br/>- Regulation (EU) 2017/745 on medical devices.<br/><b>For the UK:</b> - Medical Device Regulation (MDR) 2002 on medical devices.</p> <p><b>Expected service life:</b> 10 years.<br/><b>Contraindications for use:</b> It is not recommended that people who need to lie down while being changed use the backrest for support.</p> |
| <b>Declaration of conformity</b> |  |



# PRESSALIT®

|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS backrest, fixed height, for toilet lifter |
| <b>Item number</b> | R6408112                                       |

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®