




<p><b>Product</b></p> <p><b>Item number</b></p>	<p>PLUS backrest, height and sideways adjustable</p> <p>R6444108</p>
<p><b>Colour</b></p> <p><b>Product description</b></p>	<p>NRF:6317052</p> <p>EAN:5708590365534</p> <p>HMI: 124205</p> <p>Dark blue (RAL 5011)</p>  <p>PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters.</p> <p>PLUS backrest, 385 x 209 mm, soft feel. Height adjustable 150 mm and sideways adjustable, for mounting on a horizontal walltrack behind a wall-mounted toilet without cistern. Can be removed easily when not needed and reattached again without the use of tools. Materials: Aluminium and polyurethane foam with a plywood core. For fitting to horizontal PLUS wall track (not included).</p>
<p><b>Adjustability</b></p> <p><b>Adjustment height</b></p> <p><b>Product capacity</b></p>	<p>Height and sideways adjustable</p> <p>150 mm</p> <p>Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 33 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i></p> <p><b><i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i></b></p>
<p><b>Materials</b></p> <p><b>Ergonomy</b></p>	<p>The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer.</p> <p>Soft and slip-resistant material that has a warm feel.</p>

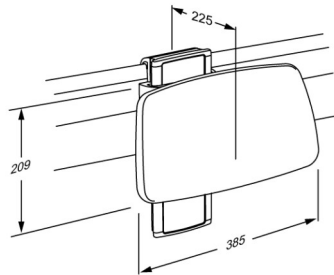

<b>Product</b>	PLUS backrest, height and sideways adjustable
<b>Item number</b>	R6444108
<b>Features</b>	For toilet without cistern
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>PLUS backrest. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Erisan OXY+ Tvättspit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)</p> <p><b>NON-DISINFECTANTS:</b> Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)</p> <p>Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>
<b>Declaration of conformity</b>	<p>CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.</p> <p><b>Expected service life:</b> 10 years. <b>Contraindications for use:</b> It is not recommended that people who need to lie down while being changed use the backrest for support.</p>



PRESSALIT®

<b>Product</b>	PLUS backrest, height and sideways adjustable
<b>Item number</b>	R6444108

---

[pressalit.com](http://pressalit.com)

---

Pressalit A/S  
Pressalitvej 1  
DK-8680 Ry  
Tel.: +45 8788 8788  
Fax: +45 8788 8789  
E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

PRESSALIT®