

PRESSALIT.

Product	CT 5000 change table, fixed height	
Item number	R8455512000	
	VVS:777540820	
	EAN:5708590376585	
	HMI: 138135	
	FDA: INT/D124320	
Color	White 10.63	
Product description	Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed hei shower and changing tables of high quality. They are designed to provide a stabl platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.	
	CT 5000 change table, lying surface in compact laminate, 27.6" x 47.2", white. Wi safety rail and body belt, foldable, fixed height, wall-mounted.	
Adjustability	Fixed mounting	
Adjustment height		
Product capacity	Maximum user weight (MUM): 331 lbs. Maximum load (SWL): 331 lbs. When the table is loaded with 331 lbs, each mounting bolt supports 115 lbs.	
	This product has been tested statically with a load of 496 lbs as the ISO 17966 standard requires testing with a safety factor 1.5.	
Materials	Powder-laquered stainless steel, lying surface in compact laminate.	
Ergonomy	The ergonomically designed safety rail has a practical lower central section, which provides better access and work environment for the carer while transferring, changing and dressing the user. Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.	



PRESSALIT.

Product	CT 5000 change table, fixed height		
Item number	R8455512000		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date.		
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as		
	long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREVention Wipes Lingettes		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the		
Declaration of conformity	surface coating may deteriorate. This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.		



PRESSALIT®

Product
Item number

CT 5000 change table, fixed height

R8455512000

Mounting in different wall types

Concrete >B25 Mounting kit, 2 lag bolts Ø10 x 70 and 2 countersunk bolts Ø10 x 80	
Solid Brick >= Mz12 Mounting kit, 2 lag bolts Ø10 x 70 and 2 countersunk bolts Ø10 x 80	
Solid Sand Lime Brick >= Hlz12 Mounting kit, 2 lag bolts Ø10 x 70 and 2 countersunk bolts Ø10 x 80	
Aerated Concrete P4 Mounting kit, 2 lag bolts Ø10 x 70 and 2 countersunk bolts Ø10 x 80	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood Mounting kit, 2 lag bolts Ø10 x 70 and 2 countersunk bolts Ø10 x 80	



PRESSALIT®

Product Item number	CT 5000 change table, fixed height R8455512000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^</u>
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^</u>
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>^</u>

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT.