

PRESSALIT.

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9662721000
Color	EAN: 5708590390000
	63.39 9
Product description	Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed heigh shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.
	MCT 3 change trolley, 27.6" x 82.7", two fold-down sides. Electrically height adjustable with wired remote control. 19.7" lifting range, from 17.7" to 37.4". Mattress to be ordered separately.
Adjustability	Height adjustable
Adjustment height	19,7 inch
Product capacity	Maximum user weight (MUM): 441 lbs. Maximum load (SWL): 452 lbs. The maximum load is to be understood as the total safe working load of the produc including both the weight of the user, and the weight of the mattress
	This product has been tested statically with a load of 1130 lbs as the IEC 60601-1 standard requires testing with a safety factor 2.5.
Speed up	Speed up: 0,67 inch/s Speed down: 0,67 inch/s
Noise level	50 dB
Materials	Plastics, compact laminate and powder-lacquered steel.



PRESSALIT.

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9662721000
Ergonomy	 The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring changing and dressing the user.
	 One hand operated safety rails allows the carer to maintain eye contact as well a physical contact with the user during the whole process, the foldable safety rails all provide easy access from all four sides of the table.
	 The cut-out grips in the safety rails enable users to assist when being turned, wh motivates physical training.
	• Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.
Features	• The trolley can be moved from room to room for easy transport and multiple user
	 Intuitive, easy-to-use drop down rails with one hand operation.
Mains connection (V)	120
Actuator (motor)	LA40 (Linak) IPX6
Control box	CA30 (Linak) IPX6
Wired remote control	HB30 (Linak) IPX6
Power consumption (maximum load)	Power consumption by maximum load: 330 W Power consumption by standby: 0,5 W
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.



PRESSALIT.

Product Item number	MCT 3 change trolley, electrically height adjustable R9662721000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREVention Wipes Lingettes
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro
	The product must be wiped down with a soft rag wrung out in clear water immedia after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
Declaration of conformity	This product is solely intended for the US and Canada, and has no EU declaration conformity.

pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

